

FREI AG Postfach 1231 79196 Kirchzarten

FREI AG

Am Fischerrain 8 | 79199 Kirchzarten  
Tel. 076 61 / 93 36 0 | Fax 076 61 / 93 36 50  
info@frei-ag.de | www.frei-ag.de

## Urgent Safety Notice

Precautionary replacement of motors on older treatment tables

**Re:** ARCUS treatment tables from FREI AG

**From:** FREI AG Aktive Reha-Systeme, Am Fischerrain 8, 79199 Kirchzarten, Germany

**Target group:** Owners, operators and distributors of the ARCUS treatment table from FREI AG

### Identification of the affected medical devices:

ARCUS treatment tables with article numbers:

Standard: 8.5100.00 ; 8.5100.50 ; 8.5100.50a  
Universal: 8.5105.00 ; 8.5105.50 ; 8.5105.50a  
Manual: 8.5110.00 ; 8.5110.50 ; 8.5110.50a  
Vojta 120cm wide: 8.5115.00 ; 8.5115.50 ; 8.5115.50a  
Vojta 100cm wide: 8.5116.00 ; 8.5116.50 ; 8.5116.50a



### Description of the problem including the determined cause:

In rare cases, the adjustment motor may fail on treatment tables with power adjustment that are older than 10 years, leading to a possible collapse of the treatment table. This is associated with a considerable risk for the patient and therapist. The causes for this possible failure are:

- Continuous load on the motor
- Natural wear due to the adjustment movements (number of cycles)
- Ageing of internal plastic parts

The damage may be indicated in advance by unusual motor noises, reduced operating comfort and possibly a slower adjustment speed.

In these cases, please stop using the treatment table immediately until it is professionally repaired.

### What steps have to be taken by the addressee?

Check the year of manufacture of all adjustment motors on your treatment tables. Motors older than 10 years should be replaced as a precaution. The age of the motor is indicated on the type plate of the motor or on the treatment table.

Example of a motor type plate:



Year of  
manufac-  
ture

If this inspection fails or cannot be carried out, please stop using these treatment tables and contact FREI AG. We are happy to assist you with any questions regarding this measure. The age of the motors and thereby the replacement interval must be checked in the course of the recurring safety inspection.

### Dissemination of the information provided here:

Please ensure that all users of the power adjustable treatment tables who need to be informed in your organisation are advised of this urgent safety notice. If you have transferred products to third parties, please pass on a copy of this information or inform the contact person below regarding the whereabouts of the treatment tables in question. Please retain this information at least until the measure has been implemented.

The Federal Institute for Drugs and Medical Devices has received a copy of this "Urgent Safety Notice".

### Contact person for further enquiries:

Julia Birkenmeier  
Telephone 07661 / 9336 625  
Julia.Birkenmeier@frei-ag.de  
FREI AG Aktive Reha-Systeme

Dominik Egetemeir  
Telephone 07661 / 9336 643  
Dominik.Egetemeir@frei-ag.de  
FREI AG Aktive Reha-Systeme

FREI AG Aktive Reha-Systeme  
Kirchzarten, Germany, May  
2018

Page 2/2