

FREI AG Postfach 1231 79196 Kirchzarten

FREI AG

Am Fischerrain 8 | 79199 Kirchzarten  
 Tel. 076 61 / 93 36 0 | Fax 076 61 / 93 36 50  
 info@frei-ag.de | www.frei-ag.de

## Urgent Safety Notice

Inspection of the height adjuster and fit of the counter nut

**Re:** FREI AG height-adjustable training table  
**From:** FREI AG Aktive Reha-Systeme, Am Fischerrain 8, 79199 Kirchzarten, Germany  
**Target group:** Owners, operators and distributors of the height adjustable training table

### Identification of the affected medical devices:

FREI AG portable and height adjustable training table – article number:

6.5550.60  
 6.5550.70  
 6.5550.71  
 6.5550.71a  
 6.5550.72



### Description of the problem including the determined cause:

The products have 2 casters on one side and 2 height adjusters on the other side. The height adjusters have to be countered, otherwise they can come unscrewed, fall off or collapse. Unsecured height adjusters can lead to considerable injuries.



Max. 20 mm between the top of the height adjuster and the round pipe

Height adjuster

Counter nut DIN 439 M10

Page 1/2

### What steps have to be taken by the addressee?

First check whether the counter nut is still installed. As the second step, please check the distance from the top of the height adjuster to the round pipe (max. 20 mm). As the third step, please use a size 17 open-ended spanner to check whether the height adjuster counter nut is tight. Viewed from above, countering is carried out counter-clockwise. Adjust the training table horizontally with the height adjusters on a level floor and tighten the counter nuts in this position.

If the counter nuts are missing, please request replacements from our contact person below, specifying the number of training tables. In case of a problem with the training device, please do not continue using it and contact us. We are happy to assist you with any questions regarding this measure.

### Dissemination of the information provided here:

Please ensure that all users of the height adjustable training table and other persons who need to be informed in your organisation are advised of this urgent safety notice. If you have transferred products to third parties, please pass on a copy of this information or inform the contact person below.

Please retain this information at least until the measure has been implemented.

The Federal Institute for Drugs and Medical Devices has received a copy of this "Urgent Safety Notice".

### Contact person:

Ms. Natalia Rommel  
Natalia.Rommel@frei-ag.de Service  
Consulting / Customer Support

FREI AG Aktive Reha-Systeme  
Kirchzarten, Germany, May 2018