

31 July, 2018

To: Surgeons / Hospitals

Subject: **URGENT MEDICAL DEVICE FIELD SAFETY NOTICE - REMOVAL**

Affected Product: Affixus Hip Fracture Nail

See Attachment 2 for a list of affected product



Zimmer Biomet is conducting a medical device Field Safety Notice (removal) for specific lots of the Affixus Hip Fracture Nail due to the set screw not being able to advance or reverse from the original position in the nail.

Risks		
Describe immediate health consequences (injuries or illness) that may result from use of or exposure to the product issue.	Most Probable	Highest Severity
	Delay in surgery < 30 mins to retrieve another part	Delay in surgery > 30 mins to retrieve another part
Describe long range health consequences (injuries or illness) that may result from use of or exposure to the product issue.	Most Probable	Highest Severity
	None	Delayed union or nonunion requiring surgical intervention

Our records indicate that you may have received one or more of the affected products. The affected units were distributed between 12 February 2018 and 25 June 2018 (manufacturer's deployment).

Surgeon / Hospital Responsibilities:

1. Review this notification and ensure that affected personnel are aware of the contents.
2. If you have affected product at your facility, assist your Zimmer Biomet sales representative and quarantine all affected product. Your Zimmer Biomet sales representative will remove the affected product from your facility.
3. Complete **Attachment 1 – Certificate of Acknowledgement** and send to fieldaction.emea@zimmerbiomet.com. This form must be returned even if you do not have affected products at your facility.
4. Retain a copy of the Certificate of Acknowledgement form with your field action records in the event of a compliance audit of your facility's documentation.
5. If you have further questions or concerns after reviewing this notice, please contact your Zimmer Biomet representative.

Other Information

This medical device Field Safety Notice was reported to all relevant Competent Authorities and the related Notified Body as required under the applicable regulations for Medical Devices as per MEDDEV 2.12-1 in Europe.


Please keep Zimmer Biomet informed of any adverse events associated with this product or any other Zimmer Biomet product by emailing winterthur.per@zimmerbiomet.com or to your local Zimmer Biomet contact.

Please be aware that the names of user facilities notified are routinely provided to the Competent Authorities for audit purposes.

The undersigned confirms that this notice has been delivered to the appropriate Regulatory Agencies.

We would like to thank you for your co-operation in advance and regret any inconveniences caused by this field action.

Sincerely,

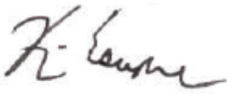

Kevin W. Escapule
Post Market Surveillance and Regulatory Compliance Director

ATTACHMENT 1

Certificate of Acknowledgement

IMMEDIATE RESPONSE REQUIRED – TIME SENSITIVE ACTION NEEDED

Affected Product: Affixus Hip Fracture Nail

Field Action Reference: ZFA 2018-00344

By signing below, I acknowledge that the required actions have been taken in accordance with this recall notice.

Hospital Facility **Surgeon** (Please check one as applicable)

Printed Name: _____ **Signature:** _____

Title: _____ **Telephone:** () _____ - _____ **Date:** ____/____/____

Facility Name: _____

Facility Address: _____

City: _____ **ZIP:** _____ **Country:** _____

Note: This form must be returned to Zimmer Biomet before this action can be considered closed for your account. It is important that you complete this form and email a copy to: fieldaction.emea@zimmerbiomet.com.

Even if you have no product to return, this form must be completed, signed and returned.

Choose the following options:

All received products were used (implanted)

Or complete the chart below for remaining products:

Product Reference	Lot Reference	Number of products returned

Comments (if needed): _____

ATTACHMENT 2 - Affected Product List

Item Number: 814309180		Item Description: Affixus® Hip Fracture Nail 125° 9 mm x 180 mm			
Lot Numbers:	017090	017100	017110	017120	063130
063190	188050	188080	188090	219720	219730
219740	219750	290520	290550	331360	371020
371060	455870	455890	530040	530060	530080
541100	559250	559260	559270	559290	688160
706780	899800	944420	530030	530090	

Item Number: 814309280		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 280 mm			
Lot Numbers:	331370	500240			

Item Number: 814309300		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 300 mm			
Lot Numbers:	017130	052890	052910		

Item Number: 814309320		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 320 mm			
Lot Numbers:	789980				

Item Number: 814309360		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 360 mm			
Lot Numbers:	050830	052950	417420		

Item Number: 814309380		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 380 mm			
Lot Numbers:	814250				

Item Number: 814309420		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 420 mm			
Lot Numbers:	151830	886880			

Item Number: 814309440		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 440 mm			
Lot Numbers:	814090	899810			

Item Number: 814309460		Item Description: Affixus® Hip Fracture Nail Right 125° 9 mm x 460 mm			
Lot Numbers:	097120	767700			

Item Number: 814311180		Item Description: Affixus® Hip Fracture Nail 125° 11 mm x 180 mm			
Lot Numbers:	017190	017200	017210	017240	017260
063270	063330	063360	063370	063400	087850
087870	097080	132740	153330	153470	153490
153510	153540	219770	219780	290570	290580
290620	500680	500710	500740	503300	503350
519650	565790	565800	585690	601380	628190

645550	645580	688170	688190	688200	688210
707670	741680	741690	741700	899820	899840
899850	899870	899880	900000		

Item Number: 814311280		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 280 mm			
Lot Numbers:	500770				

Item Number: 814311300		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 300 mm			
Lot Numbers:	132800				

Item Number: 814311320		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 320 mm			
Lot Numbers:	900040				

Item Number: 814311340		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 340 mm			
Lot Numbers:	152960	331460	456050		

Item Number: 814311360		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 360 mm			
Lot Numbers:	132900	150870	331470	773070	

Item Number: 814311380		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 380 mm			
Lot Numbers:	132920	188120	331480	688320	814070

Item Number: 814311400		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 400 mm			
Lot Numbers:	097130	245980	417450	519300	854260
645620	645650				

Item Number: 814311420		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 420 mm			
Lot Numbers:	060900	331520	417460	645670	688350

Item Number: 814311440		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 440 mm			
Lot Numbers:	050850	155250	500840	541270	559240

Item Number: 814311460		Item Description: Affixus® Hip Fracture Nail Right 125° 11 mm x 460 mm			
Lot Numbers:	017290	063450	155220	500860	

Item Number: 814313180		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 180 mm			
Lot Numbers:	060870	063550	063560	063570	114480
645700	900100	900250	944470		

Item Number: 814313260		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 260 mm	
Lot Numbers:	053670		
Item Number: 814313280		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 280 mm	
Lot Numbers:	053750		
Item Number: 814313300		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 300 mm	
Lot Numbers:	645720		
Item Number: 814313320		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 320 mm	
Lot Numbers:	331530	971020	
Item Number: 814313340		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 340 mm	
Lot Numbers:	133040	188240	645750
Item Number: 814313360		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 360 mm	
Lot Numbers:	050870	501210	
Item Number: 814313380		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 380 mm	
Lot Numbers:	482620	501270	
Item Number: 814313400		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 400 mm	
Lot Numbers:	501350		
Item Number: 814313420		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 420 mm	
Lot Numbers:	017300	097020	559190
Item Number: 814313440		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 440 mm	
Lot Numbers:	155200	502270	
Item Number: 814313460		Item Description: Affixus® Hip Fracture Nail Right 125° 13 mm x 460 mm	
Lot Numbers:	083760		
Item Number: 814409260		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 260 mm	
Lot Numbers:	971030		
Item Number: 814409280		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 280 mm	
Lot Numbers:	707700		

Item Number: 814409300		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 300 mm			
Lot Numbers:	133160				
Item Number: 814409320		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 320 mm			
Lot Numbers:	114560	133170			
Item Number: 814409340		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 340 mm			
Lot Numbers:	212940	503520		565820	
Item Number: 814409360		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 360 mm			
Lot Numbers:	097140	133190		144070	
Item Number: 814409380		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 380 mm			
Lot Numbers:	053790	155180		762720 790780 813530	
Item Number: 814409400		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 400 mm			
Lot Numbers:	051130	053810		133200 153200 813940	
Item Number: 814409420		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 420 mm			
Lot Numbers:	503570				
Item Number: 814409440		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 440 mm			
Lot Numbers:	503610				
Item Number: 814409460		Item Description: Affixus® Hip Fracture Nail Left 125° 9 mm x 460 mm			
Lot Numbers:	097160	133230			
Item Number: 814411260		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 260 mm			
Lot Numbers:	503640				
Item Number: 814411320		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 320 mm			
Lot Numbers:	155130				
Item Number: 814411340		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 340 mm			
Lot Numbers:	530180	585810			
Item Number: 814411360		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 360 mm			
Lot Numbers:	503700	646540		707750	
Item Number: 814411380		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 380 mm			
Lot Numbers:	053940	813860		886990	
Item Number: 814411400		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 400 mm			

Lot Numbers:	053980	053990	646590	768010	
Item Number: 814411420		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 420 mm			
Lot Numbers:	155120	245960	472890		
Item Number: 814411440		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 440 mm			
Lot Numbers:	151870	155110	646620	960600	
Item Number: 814411460		Item Description: Affixus® Hip Fracture Nail Left 125° 11 mm x 460 mm			
Lot Numbers:	054000	097060			
Item Number: 814413260		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 260 mm			
Lot Numbers:	814090				
Item Number: 814413280		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 280 mm			
Lot Numbers:	559130				
Item Number: 814413300		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 300 mm			
Lot Numbers:	331610				
Item Number: 814413320		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 320 mm			
Lot Numbers:	245830	331620			
Item Number: 814413340		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 340 mm			
Lot Numbers:	482730				
Item Number: 814413360		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 360 mm			
Lot Numbers:	054030	054040	312200	312210	
Item Number: 814413380		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 380 mm			
Lot Numbers:	417480	646660			
Item Number: 814413400		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 400 mm			
Lot Numbers:	054100				
Item Number: 814413420		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 420 mm			
Lot Numbers:	155080	156070	156150	482640	
Item Number: 814413460		Item Description: Affixus® Hip Fracture Nail Left 125° 13 mm x 460 mm			
Lot Numbers:	155070				
Item Number: 814509180		Item Description: Affixus® Hip Fracture Nail 130° 9 mm x 180 mm			
Lot Numbers:	054130	085710	144520	150790	156740

156750	219790	245740	245820	287290	287400
456140	482650	707780	900280	944590	944630
944640	970640				

Item Number: 814509260		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 260 mm			
Lot Numbers:	813650				

Item Number: 814509280		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 280 mm			
Lot Numbers:	768020				

Item Number: 814509300		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 300 mm			
Lot Numbers:	212980				

Item Number: 814509320		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 320 mm			
Lot Numbers:	530190	887000	971580		

Item Number: 814509340		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 340 mm			
Lot Numbers:	503940	646730			

Item Number: 814509360		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 360 mm			
Lot Numbers:	813470				

Item Number: 814509380		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 380 mm			
Lot Numbers:	628920	646790	773060		

Item Number: 814509400		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 400 mm			
Lot Numbers:	646830				

Item Number: 814509420		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 420 mm			
Lot Numbers:	153010	159190	707830		

Item Number: 814509440		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 440 mm			
Lot Numbers:	054150	054170	130740	130750	188310
519480					

Item Number: 814509460		Item Description: Affixus® Hip Fracture Nail Right 130° 9 mm x 460 mm			
Lot Numbers:	254720				

Item Number: 814511180		Item Description: Affixus® Hip Fracture Nail 130° 11 mm x 180 mm			
Lot Numbers:	072620	130760	130770	130780	130790
130800	130810	130820	130840	130850	130860
130900	130920	131030	156770	157000	188340

219800	219810	219820	219910	245570	245690
287090	287120	287130	371170	456160	519490
530110	530120	530130	558980	565870	565880
601390	707920	768050	772730	854270	901300
901330	901350	901360	901370	901380	901400
901410	944660	944720	970790	970810	970820
970830	970840	970850	970860	970870	970880
970890					

Item Number: 814511260		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 260 mm			
Lot Numbers:	791640	813460			

Item Number: 814511280		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 280 mm			
Lot Numbers:	155000	483480	646910	646930	646950

Item Number: 814511300		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 300 mm			
Lot Numbers:	558960				

Item Number: 814511320		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 320 mm			
Lot Numbers:	331650				

Item Number: 814511340		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 340 mm			
Lot Numbers:	054180	483490	519590	558900	558910
586640					

Item Number: 814511360		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 360 mm			
Lot Numbers:	107830	107850	152910	154990	482740
519610	565900	707960	762780	773410	812310
812330					

Item Number: 814511380		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 380 mm			
Lot Numbers:	054200	054210	054220		

Item Number: 814511400		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 400 mm			
Lot Numbers:	054240	054250	154980	472860	773390
798600	798610				

Item Number: 814511420		Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 420 mm			



Lot Numbers:	154960	558870	558880	628930	707860
Item Number: 814511440	Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 440 mm				
Lot Numbers:	213000	558850			
Item Number: 814511460	Item Description: Affixus® Hip Fracture Nail Right 130° 11 mm x 460 mm				
Lot Numbers:	157130	558840	710910		
Item Number: 814513180	Item Description: Affixus® Hip Fracture Nail 130° 13 mm x 180 mm				
Lot Numbers:	063590	063600	220020	245510	290720
371210	482750	482760	482770	523410	
Item Number: 814513260	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 260 mm				
Lot Numbers:	901420				
Item Number: 814513280	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 280 mm				
Lot Numbers:	213010	707970	944730		
Item Number: 814513300	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 300 mm				
Lot Numbers:	944850				
Item Number: 814513340	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 340 mm				
Lot Numbers:	017700				
Item Number: 814513380	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 380 mm				
Lot Numbers:	587670	707980	711060		
Item Number: 814513400	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 400 mm				
Lot Numbers:	152980	482780	530160	798520	798530
798540					
Item Number: 814513420	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 420 mm				
Lot Numbers:	060800	060820	397280	397290	397300
812970	901490				
Item Number: 814513440	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 440 mm				
Lot Numbers:	154920	711080			
Item Number: 814513460	Item Description: Affixus® Hip Fracture Nail Right 130° 13 mm x 460 mm				

		mm			
Lot Numbers:	558790	711090			
Item Number: 814515320		Item Description: Affixus® Hip Fracture Nail Right 130° 15 mm x 320 mm			
Lot Numbers:	154900	818440			
Item Number: 814515360		Item Description: Affixus® Hip Fracture Nail Right 130° 15 mm x 360 mm			
Lot Numbers:	154890	814740			
Item Number: 814515400		Item Description: Affixus® Hip Fracture Nail Right 130° 15 mm x 400 mm			
Lot Numbers:	364250				
Item Number: 814515440		Item Description: Affixus® Hip Fracture Nail Right 130° 15 mm x 440 mm			
Lot Numbers:	154880				
Item Number: 814609260		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 260 mm			
Lot Numbers:	157170				
Item Number: 814609280		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 280 mm			
Lot Numbers:	107860	558680	791850		
Item Number: 814609320		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 320 mm			
Lot Numbers:	055260	154870			
Item Number: 814609360		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 360 mm			
Lot Numbers:	287140				
Item Number: 814609400		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 400 mm			
Lot Numbers:	055360	055510	157290	917700	972060
Item Number: 814609420		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 420 mm			
Lot Numbers:	762840				
Item Number: 814609460		Item Description: Affixus® Hip Fracture Nail Left 130° 9 mm x 460 mm			
Lot Numbers:	055580	154820	154830		
Item Number: 814611260		Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 260 mm			

Lot Numbers:	091610	558640	629150	
---------------------	--------	--------	--------	--

Item Number: 814611280	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 280 mm			
Lot Numbers:	711150	711170		

Item Number: 814611320	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 320 mm			
Lot Numbers:	331940	558590		

Item Number: 814611340	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 340 mm				
Lot Numbers:	154800	154810	155350	629170	

Item Number: 814611360	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 360 mm				
Lot Numbers:	017730	085400	154660	154670	154680
157450	558510	565960	694270	711210	812570
812730	818350	945040			

Item Number: 814611380	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 380 mm				
Lot Numbers:	152900	154610	901540	901550	901560
901570	901580	901590	901600	565970	

Item Number: 814611400	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 400 mm				
Lot Numbers:	060780	154560	482790	530170	759190
797000	797020	812460	812540		

Item Number: 814611420	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 420 mm				
Lot Numbers:	154540	155340	712170	791890	

Item Number: 814611440	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 440 mm			
Lot Numbers:	558490	759400		

Item Number: 814611460	Item Description: Affixus® Hip Fracture Nail Left 130° 11 mm x 460 mm			
Lot Numbers:	171830	712290		

Item Number: 814613260	Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 260 mm			
Lot Numbers:	171820	972190		

Item Number: 814613280	Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 280 mm			
Lot Numbers:	688510	763000		

Item Number: 814613300		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 300 mm			
Lot Numbers:	818430	960660			
Item Number: 814613320		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 320 mm			
Lot Numbers:	245390	287180			
Item Number: 814613340		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 340 mm			
Lot Numbers:	331960				
Item Number: 814613360		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 360 mm			
Lot Numbers:	816070				
Item Number: 814613380		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 380 mm			
Lot Numbers:	712320				
Item Number: 814613400		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 400 mm			
Lot Numbers:	055730	060770	154510	558410	712370
Item Number: 814613420		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 420 mm			
Lot Numbers:	154480				
Item Number: 814613440		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 440 mm			
Lot Numbers:	157490	482830			
Item Number: 814613460		Item Description: Affixus® Hip Fracture Nail Left 130° 13 mm x 460 mm			
Lot Numbers:	055760	154460	503740	558390	712400
Item Number: 814615360		Item Description: Affixus® Hip Fracture Nail Left 130° 15 mm x 360 mm			
Lot Numbers:	154450				