

Urgent Field Safety Notice

August, 2019

Dear Healthcare Provider:

Problem Description

Baxter Healthcare has identified the potential for certain PrisMax devices to develop cracks in the blood pump raceway over time (refer to figure 1). The cracks are caused by varying material thickness in this specific area of the front panel. To ensure high quality, and to avoid any problems with the front panel over time, the front panel will be replaced on all potentially impacted units listed in the affected product table below.



Figure 1: Pictures of PrisMax device and cracks in the blood pump raceway

A crack in the blood pump raceway is likely to cause insufficient occlusion of the blood pump. If this occurs before treatment, the device will no longer pass the prime test. If this occurs during treatment, therapy will be interrupted. A device affected by this failure will not be usable until an off-site repair by Baxter Service is completed.

Baxter has implemented corrective actions to resolve this issue in newly manufactured PrisMax devices.

Affected Product

Product Code	Product Description	Serial Numbers
955558	PrisMax, V2, ROW	Refer to Attachment A: Affected Product Table

**Hazard
Involved**

This issue may result in interruption of therapy, delay in therapy, or blood loss due to non-restitution of blood in the extracorporeal circuit; however, adverse health consequences are not likely. There have been no reports of serious injury associated with this issue.

**Actions to be
Taken by
Customers**

1. Operators may continue to safely use affected devices that have not exhibited failures of the blood pump raceway. If the device is not performing as intended as a result of this issue, the user will be alerted by Alarm T0850, Alarm T0525, or Alarm T0602.
2. A local Baxter service representative will contact your facility to determine the correction plan and schedule the off-site device repair. Loaner devices will be provided as needed. If you experience an alarm, please contact Baxter Technical Service.
3. Complete the enclosed Baxter Customer Reply Form and return it to Baxter by faxing it or scanning and e-mailing it. Returning the customer reply form promptly will confirm your receipt of this notification and prevent you from receiving repeat notices.
4. If you distribute this product to other facilities or departments within your institution, please forward a copy of this communication to them.

**Further
information
and support**

For general questions regarding this communication, contact Baxter.

The local Ministry of Health (MoH) has been notified of this action.

We apologize for any inconvenience this may cause you and your staff.

Sincerely,

Baxter Healthcare Corporation

Affected product table

Product Code 955558; PrisMax, V2, ROW

Date Range Produced: 11-Oct-2018 to 26-Apr-2019

Count	Serial Number	UDI Number
1	100265	(01)07332414124359 (21) 100265
2	100266	(01)07332414124359 (21) 100266
3	100267	(01)07332414124359 (21) 100267
4	100268	(01)07332414124359 (21) 100268
5	100269	(01)07332414124359 (21) 100269
6	100270	(01)07332414124359 (21) 100270
7	100271	(01)07332414124359 (21) 100271
8	100272	(01)07332414124359 (21) 100272
9	100273	(01)07332414124359 (21) 100273
10	100274	(01)07332414124359 (21) 100274
11	100276	(01)07332414124359 (21) 100276
12	100277	(01)07332414124359 (21) 100277
13	100278	(01)07332414124359 (21) 100278
14	100279	(01)07332414124359 (21) 100279
15	100300	(01)07332414124359 (21) 100300
16	100301	(01)07332414124359 (21) 100301
17	100302	(01)07332414124359 (21) 100302
18	100303	(01)07332414124359 (21) 100303
19	100304	(01)07332414124359 (21) 100304
20	100316	(01)07332414124359 (21) 100316
21	100317	(01)07332414124359 (21) 100317
22	100319	(01)07332414124359 (21) 100319
23	100320	(01)07332414124359 (21) 100320
24	100331	(01)07332414124359 (21) 100331
25	100332	(01)07332414124359 (21) 100332
26	100333	(01)07332414124359 (21) 100333
27	100334	(01)07332414124359 (21) 100334
28	100335	(01)07332414124359 (21) 100335
29	100336	(01)07332414124359 (21) 100336
30	100338	(01)07332414124359 (21) 100338
31	100339	(01)07332414124359 (21) 100339
32	100340	(01)07332414124359 (21) 100340
33	100351	(01)07332414124359 (21) 100351
34	100352	(01)07332414124359 (21) 100352
35	100354	(01)07332414124359 (21) 100354
36	100355	(01)07332414124359 (21) 100355
37	100371	(01)07332414124359 (21) 100371
38	100372	(01)07332414124359 (21) 100372
39	100373	(01)07332414124359 (21) 100373
40	100374	(01)07332414124359 (21) 100374
41	100375	(01)07332414124359 (21) 100375
42	100376	(01)07332414124359 (21) 100376
43	100377	(01)07332414124359 (21) 100377
44	100378	(01)07332414124359 (21) 100378
45	100380	(01)07332414124359 (21) 100380
46	100381	(01)07332414124359 (21) 100381

Affected product table

47	100382	(01)07332414124359 (21) 100382
48	100384	(01)07332414124359 (21) 100384
49	100385	(01)07332414124359 (21) 100385
50	100386	(01)07332414124359 (21) 100386
51	100387	(01)07332414124359 (21) 100387
52	100389	(01)07332414124359 (21) 100389
53	100390	(01)07332414124359 (21) 100390
54	100406	(01)07332414124359 (21) 100406
55	100409	(01)07332414124359 (21) 100409
56	100410	(01)07332414124359 (21) 100410
57	100411	(01)07332414124359 (21) 100411
58	100412	(01)07332414124359 (21) 100412
59	100413	(01)07332414124359 (21) 100413
60	100414	(01)07332414124359 (21) 100414
61	100432	(01)07332414124359 (21) 100432
62	100433	(01)07332414124359 (21) 100433
63	100443	(01)07332414124359 (21) 100443
64	100444	(01)07332414124359 (21) 100444
65	100447	(01)07332414124359 (21) 100447
66	100462	(01)07332414124359 (21) 100462
67	100463	(01)07332414124359 (21) 100463
68	100464	(01)07332414124359 (21) 100464
69	100465	(01)07332414124359 (21) 100465
70	100472	(01)07332414124359 (21) 100472
71	100474	(01)07332414124359 (21) 100474
72	100475	(01)07332414124359 (21) 100475
73	100476	(01)07332414124359 (21) 100476
74	100479	(01)07332414124359 (21) 100479
75	100480	(01)07332414124359 (21) 100480
76	100481	(01)07332414124359 (21) 100481
77	100483	(01)07332414124359 (21) 100483
78	100484	(01)07332414124359 (21) 100484
79	100485	(01)07332414124359 (21) 100485
80	100506	(01)07332414124359 (21) 100506
81	100508	(01)07332414124359 (21) 100508
82	100510	(01)07332414124359 (21) 100510
83	100511	(01)07332414124359 (21) 100511
84	100512	(01)07332414124359 (21) 100512
85	100513	(01)07332414124359 (21) 100513
86	100514	(01)07332414124359 (21) 100514
87	100516	(01)07332414124359 (21) 100516
88	100518	(01)07332414124359 (21) 100518
89	100519	(01)07332414124359 (21) 100519
90	100520	(01)07332414124359 (21) 100520
91	100521	(01)07332414124359 (21) 100521
92	100522	(01)07332414124359 (21) 100522
93	100524	(01)07332414124359 (21) 100524
94	100525	(01)07332414124359 (21) 100525
95	100526	(01)07332414124359 (21) 100526
96	100527	(01)07332414124359 (21) 100527

Affected product table

97	100528	(01)07332414124359 (21) 100528
98	100529	(01)07332414124359 (21) 100529
99	100530	(01)07332414124359 (21) 100530
100	100556	(01)07332414124359 (21) 100556
101	100558	(01)07332414124359 (21) 100558
102	100559	(01)07332414124359 (21) 100559
103	100560	(01)07332414124359 (21) 100560
104	100561	(01)07332414124359 (21) 100561
105	100562	(01)07332414124359 (21) 100562
106	100563	(01)07332414124359 (21) 100563
107	100564	(01)07332414124359 (21) 100564
108	100565	(01)07332414124359 (21) 100565
109	100566	(01)07332414124359 (21) 100566
110	100567	(01)07332414124359 (21) 100567
111	100568	(01)07332414124359 (21) 100568
112	100570	(01)07332414124359 (21) 100570
113	100571	(01)07332414124359 (21) 100571
114	100573	(01)07332414124359 (21) 100573
115	100576	(01)07332414124359 (21) 100576
116	100577	(01)07332414124359 (21) 100577
117	100578	(01)07332414124359 (21) 100578
118	100608	(01)07332414124359 (21) 100608
119	100609	(01)07332414124359 (21) 100609
120	100611	(01)07332414124359 (21) 100611
121	100612	(01)07332414124359 (21) 100612
122	100614	(01)07332414124359 (21) 100614
123	100615	(01)07332414124359 (21) 100615
124	100616	(01)07332414124359 (21) 100616
125	100617	(01)07332414124359 (21) 100617
126	100618	(01)07332414124359 (21) 100618
127	100619	(01)07332414124359 (21) 100619
128	100620	(01)07332414124359 (21) 100620
129	100621	(01)07332414124359 (21) 100621
130	100622	(01)07332414124359 (21) 100622
131	100623	(01)07332414124359 (21) 100623
132	100624	(01)07332414124359 (21) 100624
133	100625	(01)07332414124359 (21) 100625
134	100626	(01)07332414124359 (21) 100626
135	100627	(01)07332414124359 (21) 100627
136	100628	(01)07332414124359 (21) 100628
137	100629	(01)07332414124359 (21) 100629
138	100656	(01)07332414124359 (21) 100656
139	100657	(01)07332414124359 (21) 100657
140	100658	(01)07332414124359 (21) 100658
141	100659	(01)07332414124359 (21) 100659
142	100660	(01)07332414124359 (21) 100660
143	100661	(01)07332414124359 (21) 100661
144	100662	(01)07332414124359 (21) 100662
145	100664	(01)07332414124359 (21) 100664
146	100665	(01)07332414124359 (21) 100665

Affected product table

147	100666	(01)07332414124359 (21) 100666
148	100667	(01)07332414124359 (21) 100667
149	100668	(01)07332414124359 (21) 100668
150	100669	(01)07332414124359 (21) 100669
151	100670	(01)07332414124359 (21) 100670
152	100671	(01)07332414124359 (21) 100671
153	100672	(01)07332414124359 (21) 100672
154	100673	(01)07332414124359 (21) 100673
155	100674	(01)07332414124359 (21) 100674
156	100675	(01)07332414124359 (21) 100675
157	100676	(01)07332414124359 (21) 100676
158	100677	(01)07332414124359 (21) 100677
159	100678	(01)07332414124359 (21) 100678
160	100679	(01)07332414124359 (21) 100679
161	100681	(01)07332414124359 (21) 100681
162	100682	(01)07332414124359 (21) 100682
163	100687	(01)07332414124359 (21) 100687
164	100688	(01)07332414124359 (21) 100688
165	100689	(01)07332414124359 (21) 100689
166	100690	(01)07332414124359 (21) 100690
167	100692	(01)07332414124359 (21) 100692
168	100693	(01)07332414124359 (21) 100693
169	100696	(01)07332414124359 (21) 100696
170	100697	(01)07332414124359 (21) 100697
171	100698	(01)07332414124359 (21) 100698
172	100699	(01)07332414124359 (21) 100699
173	100700	(01)07332414124359 (21) 100700
174	100701	(01)07332414124359 (21) 100701
175	100702	(01)07332414124359 (21) 100702
176	100703	(01)07332414124359 (21) 100703
177	100704	(01)07332414124359 (21) 100704
178	100706	(01)07332414124359 (21) 100706
179	100708	(01)07332414124359 (21) 100708
180	100709	(01)07332414124359 (21) 100709
181	100711	(01)07332414124359 (21) 100711
182	100712	(01)07332414124359 (21) 100712
183	100713	(01)07332414124359 (21) 100713
184	100715	(01)07332414124359 (21) 100715
185	100717	(01)07332414124359 (21) 100717
186	100718	(01)07332414124359 (21) 100718
187	100720	(01)07332414124359 (21) 100720
188	100721	(01)07332414124359 (21) 100721
189	100722	(01)07332414124359 (21) 100722
190	100723	(01)07332414124359 (21) 100723
191	100724	(01)07332414124359 (21) 100724
192	100725	(01)07332414124359 (21) 100725
193	100726	(01)07332414124359 (21) 100726
194	100727	(01)07332414124359 (21) 100727
195	100728	(01)07332414124359 (21) 100728
196	100730	(01)07332414124359 (21) 100730

Affected product table

197	100733	(01)07332414124359 (21) 100733
198	100734	(01)07332414124359 (21) 100734
199	100735	(01)07332414124359 (21) 100735
200	100736	(01)07332414124359 (21) 100736
201	100737	(01)07332414124359 (21) 100737
202	100738	(01)07332414124359 (21) 100738
203	100740	(01)07332414124359 (21) 100740
204	100741	(01)07332414124359 (21) 100741
205	100743	(01)07332414124359 (21) 100743
206	100744	(01)07332414124359 (21) 100744
207	100745	(01)07332414124359 (21) 100745
208	100746	(01)07332414124359 (21) 100746
209	100749	(01)07332414124359 (21) 100749
210	100832	(01)07332414124359 (21) 100832
211	100834	(01)07332414124359 (21) 100834
212	100835	(01)07332414124359 (21) 100835
213	100838	(01)07332414124359 (21) 100838
214	100839	(01)07332414124359 (21) 100839
215	100842	(01)07332414124359 (21) 100842
216	100843	(01)07332414124359 (21) 100843
217	100853	(01)07332414124359 (21) 100853
218	100855	(01)07332414124359 (21) 100855
219	100907	(01)07332414124359 (21) 100907
220	100908	(01)07332414124359 (21) 100908
221	100909	(01)07332414124359 (21) 100909
222	100920	(01)07332414124359 (21) 100920