Medline International Germany GmbH
Medline-Straße 1-3
D-47533 Kleve
Tel: +49 (0) 2821 – 7510 – 0
Fax: +49 (0) 2821 – 7510 – 7802

Regulatory Affairs
Quality Department
gmb-eu-ra-kleve@medline.com
Tel: +49 (0) 2821 – 7510 – 7528
Fax: +49 (0) 2821 – 7510 – 7804

www.medline.com/de

FIELD SAFETY NOTICE

Medical Device Safety Advisory Notice

Kleve, December 9th, 2019

For the attention of: the Pharmacist responsible for medical device vigilance and the Biomedical Engineering Department.

SECURITY INFORMATION of Fluid collection pouches with graduations

Medline reference: FSN-19/08
MoH reference:
Description: Fluid collection pouches with graduations used on maternity drapes
Product Codes concerned: See annex 1

Dear Customer,

This letter is to inform you on a security information concerning fluid collection pouches with graduations included in the product references listed in [Annex 1](#).

Issues:

An incident was reported to Medline International France concerning a graduated fluid collection pouch used on a maternity drape. The graduation scale on the fluid collection pouches are approximate and in the case of this incident the healthcare professional did not make a correct diagnostic. This would lead to a possible underestimated quantity of fluids loss (blood in particularly), which can lead to inappropriate care and discomfort.

Medline International France, would like to inform you that fluid collection pouches are not measuring devices, they are used to provide healthcare professional an indication on the loss of patient fluids. In any case, external calibrated weighing scales should be used for a precise reading.

To reduce the risk of miss-use, Medline has placed a warning label on products containing graduated fluid collection pouch. This will accompany the concerned products until the information can be included in an Instruction For Use (IFU) – see [Annex 2](#).

The graduation scale is to be used as an indication; however, Medline is in progress of modifying the graduation scale to improve the precision.

Photo of the graduated fluid collection pouch concerned:

Medline International Germany GmbH
Medline-Straße 1-3
D-47533 Kleve
Tel: +49 (0) 2821 – 7510 – 0
Fax: +49 (0) 2821 – 7510 – 7802

Regulatory Affairs
Quality Department
gmb-eu-ra-kleve@medline.com
Tel: +49 (0) 2821 – 7510 – 7528
Fax: +49 (0) 2821 – 7510 – 7804

www.medline.com/de

Actions to be taken:

1. Urgently check your stock and promptly put in quarantine the concerned products listed in Annex 1.
2. Complete the acknowledgement form by either fax or email as soon as possible, but not later than January 17th, 2020 and indicate the quantity of products in stock, to receive the necessary "warning stickers".
3. Put a "warning sticker" on each concerned product in your stock, release the labelled products and refer to Annex 2 for the IFU.

The relevant competent authorities are informed of this safety notice.

We apologize for the inconvenience caused.

Yours Sincerely,
Kenneth Smith
International Quality and Regulatory Affairs Manager.

PS: This safety information is only addressed to facilities that had received the concerned products.

Medline International Germany GmbH
Medline-Straße 1-3
D-47533 Kleve
Tel: +49 (0) 2821 – 7510 – 0
Fax: +49 (0) 2821 – 7510 – 7802

Regulatory Affairs
Quality Department
gmb-eu-ra-kleve@medline.com
Tel: +49 (0) 2821 – 7510 – 7528
Fax: +49 (0) 2821 – 7510 – 7804

www.medline.com/de

**Acknowledgement receipt to fax to the following fax number: +49 2821 7510 7822
or send by email to: gmb-eu-ra-kleve@medline.com**

Medline reference: FSN-19/08

Please complete and send back the enclosed acknowledgment form by either fax or email to Medline as soon as possible, but no later than **January 17th, 2020**.

Medline International France references concerned by this notification are listed in [Annex 1](#).

Quantity of warning labels required: _____

I have read and understood the security information provided by Medline and I acknowledge receipt of the FSN-19/08.

I also agree to further distribute and communicate this important information within my facility as required.

If you distribute this product to other facilities or departments within your institution, please forward a copy of this communication to them.

If you are a dealer, wholesaler, distributor/reseller, or original equipment manufacturer (OEM) that distributed any affected products to other facilities, please distribute this notification to customers and confirm us that your customers have been notified.

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Medline International France SAS,