Unser Zeichen: MNE/19-194

Bobigny, 8. Januar 2020

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INFORMATION REGARDING THE CABLE

The PETERS SURGICAL monopolar connection cable is exclusively designed for monopolar laparoscopic surgical instruments. The cable should only be used by qualified practitioners having gained due experience thereof.

- **Maximum assigned voltage: 5000 volts at peak value.**

↓ Make sure that the voltage assigned to any accessory used in conjunction with a high-frequency (HF) generator is superior to that of the generator itself (power setting, mode) (see IEC 60601-2-2 Standard). Please refer to the HF generator Instructions' Manual.

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▲ **INDICATIONS OF USE**

The PETERS SURGICAL monopolar connection cable is exclusively designed for monopolar laparoscopic surgical instruments. The cable should only be used by qualified practitioners having gained due experience thereof.

- **Maximum assigned voltage: 2000 volts at peak value.**

↓ Ensure compatibility with all other associated instruments; in particular, the connections and voltage supplied by the HF generator should be lower than the assigned voltages of the accessories being used in combination (see IEC 60601-2-2). Check the instruction for use of the cable and of the HF generator regarding its operating, as well as its power and mode settings.

NE: Only surgical monopolar cables may be connected to PETERS SURGICAL monopolar instruments

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