

2020-09-18

URGENT - FIELD SAFETY NOTICE

Type of Action	Replacement of the PCBA Connectors within affected BMU40
Subject:	FSCA-2020-07-30 BMU40 Replacement of PCBA Connector
Affected Product:	70104.0852 - Blood Monitoring Unit BMU 40
Affected Serial Numbers:	From 90002001 to 90002313 (see detailed information in Annex I – List of affected products)

Dear valued customer,

The medical device Blood Monitoring Unit BMU 40 is designed to monitor blood parameters during cardiopulmonary bypass (CPB) or similar procedures with extracorporeal circulation, which require continuous monitoring of the arterial and/or venous blood parameters: partial pressure of oxygen (pO₂), temperature (T_{Art} and T_{Ven}), oxygen saturation (SO₂), hemoglobin (Hb) and hematocrit (Hct). Oxygen consumption (VO₂) can also be calculated. Blood flow (Q_{Blood}) can be entered manually or values can be received from a connected heart-lung machine.

It has become known to Maquet Cardiopulmonary GmbH that the printed circuit board (PCBA Connector) of the BMU40 was designed with a short creepage distance. This PCBA Connector is positioned in the inner and backside of the metal enclosure of the BMU40.

Taking into consideration the product specifications, the mitigating design factors, as well as the factors that may contribute to product risk, the following potential health consequences (harms) could occur:

- Potential of electric shock
- Possible absence or potential loss of patient data during device replacement
- Procedural/use delay during device replacement
- User inconvenience during device replacement

The following

- design factors,
- the customary use of standard PPE by a user of the BMU40 (viz. nitrile gloves),
- functional healthcare facility protections,
- proper grounding of any accessory equipment connected to the BMU40, and
- IFU recommendations (e.g. a properly connected BMU40 to protective ground/earth, the use of equipotential bonding, noncontact of the device and the patient simultaneously, etc.),

would, more than likely, greatly attenuate the risk/ potential of electric shock passing through the BMU40 housing (backside) to a user.

Maquet Cardiopulmonary GmbH has not received any reports of serious injuries or death due to the malfunction of the PCBA Connector.

Based on the associated risk to the PCBA Connector with a shorter creepage distance, a general decommissioning of the affected BMU40 is therefore not required.

Corrective Action:

- The PCBA Connector within the affected BMU40 with the serial numbers mentioned above will be replaced by a new one.
- With a pending replacement of the PCBA Connector, the facility may continue to use the BMU40 by:
 - using the equipotential bonding connector on the backside of the BMU40, and
 - not connecting external devices to the BMU40 serial ports (Com 1 /Com 2) with an operational/ communication voltage outside the range of $\pm 25\text{Vdc}$

Advice on action to be taken by the User

- Your current stock may include products affected by this action.
- The PCBA Connector in the affected BMU40's shall be replaced by the new PCBA Connector as soon as possible, at the latest during the next annual preventive maintenance.
- Your local Getinge representative will contact you to arrange the replacement of the PCBA Connector of your BMU40.
- If you have an affected BMU40 unit, duly complete the enclosed Letter of Acknowledgement Customer and return it to your local Getinge representative as soon as possible by mentioning FSCA-2020-07-30 as a reference.

Referenced documents/ attachments:

- Letter of Acknowledgement Customer
- Annex I – List of affected products

Transmission of the Field Safety Notice:

- This notice needs to be passed on to all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred.
- Please transfer this notice to other organizations on which the action could have an impact.
- Please maintain awareness of the notice and resulting actions for an appropriate period to ensure effectiveness of the corrective action.

We apologize for any inconvenience this may cause you and we will do our utmost to carry through this action as swiftly as possible.

As required, we will provide this notification to the necessary Regulatory Agencies.

Should you have questions or require additional information, please contact your local Getinge representative.

Sincerely,

Managing Director `sig,bio=1`

Safety Officer `sig,bio=1`

Maquet Cardiopulmonary GmbH
Kehler Str. 31
76437 Rastatt
GERMANY

This Annex I is considered as a supplementary attachment to the Field Safety Notice of FSCA-2020-07-30

Below are listed all serial numbers of **Blood Monitoring Unit BMU 40** which are affected by this FSCA-2020-07-30.

No.	Material No.	Material Description	Serial No.
1	70104.0852	Blood Monitoring Unit BMU 40	90002001
2	70104.0852	Blood Monitoring Unit BMU 40	90002002
3	70104.0852	Blood Monitoring Unit BMU 40	90002003
4	70104.0852	Blood Monitoring Unit BMU 40	90002004
5	70104.0852	Blood Monitoring Unit BMU 40	90002005
6	70104.0852	Blood Monitoring Unit BMU 40	90002006
7	70104.0852	Blood Monitoring Unit BMU 40	90002007
8	70104.0852	Blood Monitoring Unit BMU 40	90002008
9	70104.0852	Blood Monitoring Unit BMU 40	90002009
10	70104.0852	Blood Monitoring Unit BMU 40	90002010
11	70104.0852	Blood Monitoring Unit BMU 40	90002011
12	70104.0852	Blood Monitoring Unit BMU 40	90002012
13	70104.0852	Blood Monitoring Unit BMU 40	90002013
14	70104.0852	Blood Monitoring Unit BMU 40	90002014
15	70104.0852	Blood Monitoring Unit BMU 40	90002015
16	70104.0852	Blood Monitoring Unit BMU 40	90002016
17	70104.0852	Blood Monitoring Unit BMU 40	90002017
18	70104.0852	Blood Monitoring Unit BMU 40	90002018
19	70104.0852	Blood Monitoring Unit BMU 40	90002019
20	70104.0852	Blood Monitoring Unit BMU 40	90002020
21	70104.0852	Blood Monitoring Unit BMU 40	90002021
22	70104.0852	Blood Monitoring Unit BMU 40	90002022
23	70104.0852	Blood Monitoring Unit BMU 40	90002023
24	70104.0852	Blood Monitoring Unit BMU 40	90002024
25	70104.0852	Blood Monitoring Unit BMU 40	90002025
26	70104.0852	Blood Monitoring Unit BMU 40	90002026
27	70104.0852	Blood Monitoring Unit BMU 40	90002027
28	70104.0852	Blood Monitoring Unit BMU 40	90002028
29	70104.0852	Blood Monitoring Unit BMU 40	90002029
30	70104.0852	Blood Monitoring Unit BMU 40	90002030
31	70104.0852	Blood Monitoring Unit BMU 40	90002031
32	70104.0852	Blood Monitoring Unit BMU 40	90002032
33	70104.0852	Blood Monitoring Unit BMU 40	90002033
34	70104.0852	Blood Monitoring Unit BMU 40	90002034
35	70104.0852	Blood Monitoring Unit BMU 40	90002035
36	70104.0852	Blood Monitoring Unit BMU 40	90002036

37	70104.0852	Blood Monitoring Unit BMU 40	90002037
38	70104.0852	Blood Monitoring Unit BMU 40	90002038
39	70104.0852	Blood Monitoring Unit BMU 40	90002039
40	70104.0852	Blood Monitoring Unit BMU 40	90002040
41	70104.0852	Blood Monitoring Unit BMU 40	90002041
42	70104.0852	Blood Monitoring Unit BMU 40	90002042
43	70104.0852	Blood Monitoring Unit BMU 40	90002043
44	70104.0852	Blood Monitoring Unit BMU 40	90002044
45	70104.0852	Blood Monitoring Unit BMU 40	90002045
46	70104.0852	Blood Monitoring Unit BMU 40	90002046
47	70104.0852	Blood Monitoring Unit BMU 40	90002047
48	70104.0852	Blood Monitoring Unit BMU 40	90002048
49	70104.0852	Blood Monitoring Unit BMU 40	90002049
50	70104.0852	Blood Monitoring Unit BMU 40	90002050
51	70104.0852	Blood Monitoring Unit BMU 40	90002051
52	70104.0852	Blood Monitoring Unit BMU 40	90002052
53	70104.0852	Blood Monitoring Unit BMU 40	90002053
54	70104.0852	Blood Monitoring Unit BMU 40	90002054
55	70104.0852	Blood Monitoring Unit BMU 40	90002055
56	70104.0852	Blood Monitoring Unit BMU 40	90002056
57	70104.0852	Blood Monitoring Unit BMU 40	90002057
58	70104.0852	Blood Monitoring Unit BMU 40	90002058
59	70104.0852	Blood Monitoring Unit BMU 40	90002059
60	70104.0852	Blood Monitoring Unit BMU 40	90002060
61	70104.0852	Blood Monitoring Unit BMU 40	90002061
62	70104.0852	Blood Monitoring Unit BMU 40	90002062
63	70104.0852	Blood Monitoring Unit BMU 40	90002063
64	70104.0852	Blood Monitoring Unit BMU 40	90002064
65	70104.0852	Blood Monitoring Unit BMU 40	90002065
66	70104.0852	Blood Monitoring Unit BMU 40	90002066
67	70104.0852	Blood Monitoring Unit BMU 40	90002067
68	70104.0852	Blood Monitoring Unit BMU 40	90002068
69	70104.0852	Blood Monitoring Unit BMU 40	90002069
70	70104.0852	Blood Monitoring Unit BMU 40	90002070
71	70104.0852	Blood Monitoring Unit BMU 40	90002071
72	70104.0852	Blood Monitoring Unit BMU 40	90002072
73	70104.0852	Blood Monitoring Unit BMU 40	90002073
74	70104.0852	Blood Monitoring Unit BMU 40	90002074
75	70104.0852	Blood Monitoring Unit BMU 40	90002075
76	70104.0852	Blood Monitoring Unit BMU 40	90002076
77	70104.0852	Blood Monitoring Unit BMU 40	90002077
78	70104.0852	Blood Monitoring Unit BMU 40	90002078
79	70104.0852	Blood Monitoring Unit BMU 40	90002079
80	70104.0852	Blood Monitoring Unit BMU 40	90002080

81	70104.0852	Blood Monitoring Unit BMU 40	90002081
82	70104.0852	Blood Monitoring Unit BMU 40	90002082
83	70104.0852	Blood Monitoring Unit BMU 40	90002083
84	70104.0852	Blood Monitoring Unit BMU 40	90002084
85	70104.0852	Blood Monitoring Unit BMU 40	90002085
86	70104.0852	Blood Monitoring Unit BMU 40	90002086
87	70104.0852	Blood Monitoring Unit BMU 40	90002087
88	70104.0852	Blood Monitoring Unit BMU 40	90002088
89	70104.0852	Blood Monitoring Unit BMU 40	90002089
90	70104.0852	Blood Monitoring Unit BMU 40	90002090
91	70104.0852	Blood Monitoring Unit BMU 40	90002091
92	70104.0852	Blood Monitoring Unit BMU 40	90002092
93	70104.0852	Blood Monitoring Unit BMU 40	90002093
94	70104.0852	Blood Monitoring Unit BMU 40	90002094
95	70104.0852	Blood Monitoring Unit BMU 40	90002095
96	70104.0852	Blood Monitoring Unit BMU 40	90002096
97	70104.0852	Blood Monitoring Unit BMU 40	90002097
98	70104.0852	Blood Monitoring Unit BMU 40	90002098
99	70104.0852	Blood Monitoring Unit BMU 40	90002099
100	70104.0852	Blood Monitoring Unit BMU 40	90002100
101	70104.0852	Blood Monitoring Unit BMU 40	90002101
102	70104.0852	Blood Monitoring Unit BMU 40	90002102
103	70104.0852	Blood Monitoring Unit BMU 40	90002103
104	70104.0852	Blood Monitoring Unit BMU 40	90002104
105	70104.0852	Blood Monitoring Unit BMU 40	90002105
106	70104.0852	Blood Monitoring Unit BMU 40	90002106
107	70104.0852	Blood Monitoring Unit BMU 40	90002107
108	70104.0852	Blood Monitoring Unit BMU 40	90002108
109	70104.0852	Blood Monitoring Unit BMU 40	90002109
110	70104.0852	Blood Monitoring Unit BMU 40	90002110
111	70104.0852	Blood Monitoring Unit BMU 40	90002111
112	70104.0852	Blood Monitoring Unit BMU 40	90002112
113	70104.0852	Blood Monitoring Unit BMU 40	90002113
114	70104.0852	Blood Monitoring Unit BMU 40	90002114
115	70104.0852	Blood Monitoring Unit BMU 40	90002115
116	70104.0852	Blood Monitoring Unit BMU 40	90002116
117	70104.0852	Blood Monitoring Unit BMU 40	90002117
118	70104.0852	Blood Monitoring Unit BMU 40	90002118
119	70104.0852	Blood Monitoring Unit BMU 40	90002119
120	70104.0852	Blood Monitoring Unit BMU 40	90002120
121	70104.0852	Blood Monitoring Unit BMU 40	90002121
122	70104.0852	Blood Monitoring Unit BMU 40	90002122
123	70104.0852	Blood Monitoring Unit BMU 40	90002123
124	70104.0852	Blood Monitoring Unit BMU 40	90002124

125	70104.0852	Blood Monitoring Unit BMU 40	90002125
126	70104.0852	Blood Monitoring Unit BMU 40	90002126
127	70104.0852	Blood Monitoring Unit BMU 40	90002127
128	70104.0852	Blood Monitoring Unit BMU 40	90002128
129	70104.0852	Blood Monitoring Unit BMU 40	90002129
130	70104.0852	Blood Monitoring Unit BMU 40	90002130
131	70104.0852	Blood Monitoring Unit BMU 40	90002131
132	70104.0852	Blood Monitoring Unit BMU 40	90002132
133	70104.0852	Blood Monitoring Unit BMU 40	90002133
134	70104.0852	Blood Monitoring Unit BMU 40	90002134
135	70104.0852	Blood Monitoring Unit BMU 40	90002135
136	70104.0852	Blood Monitoring Unit BMU 40	90002136
137	70104.0852	Blood Monitoring Unit BMU 40	90002137
138	70104.0852	Blood Monitoring Unit BMU 40	90002138
139	70104.0852	Blood Monitoring Unit BMU 40	90002139
140	70104.0852	Blood Monitoring Unit BMU 40	90002140
141	70104.0852	Blood Monitoring Unit BMU 40	90002141
142	70104.0852	Blood Monitoring Unit BMU 40	90002142
143	70104.0852	Blood Monitoring Unit BMU 40	90002143
144	70104.0852	Blood Monitoring Unit BMU 40	90002144
145	70104.0852	Blood Monitoring Unit BMU 40	90002145
146	70104.0852	Blood Monitoring Unit BMU 40	90002146
147	70104.0852	Blood Monitoring Unit BMU 40	90002147
148	70104.0852	Blood Monitoring Unit BMU 40	90002148
149	70104.0852	Blood Monitoring Unit BMU 40	90002149
150	70104.0852	Blood Monitoring Unit BMU 40	90002150
151	70104.0852	Blood Monitoring Unit BMU 40	90002151
152	70104.0852	Blood Monitoring Unit BMU 40	90002152
153	70104.0852	Blood Monitoring Unit BMU 40	90002153
154	70104.0852	Blood Monitoring Unit BMU 40	90002154
155	70104.0852	Blood Monitoring Unit BMU 40	90002155
156	70104.0852	Blood Monitoring Unit BMU 40	90002156
157	70104.0852	Blood Monitoring Unit BMU 40	90002157
158	70104.0852	Blood Monitoring Unit BMU 40	90002158
159	70104.0852	Blood Monitoring Unit BMU 40	90002159
160	70104.0852	Blood Monitoring Unit BMU 40	90002160
161	70104.0852	Blood Monitoring Unit BMU 40	90002161
162	70104.0852	Blood Monitoring Unit BMU 40	90002162
163	70104.0852	Blood Monitoring Unit BMU 40	90002163
164	70104.0852	Blood Monitoring Unit BMU 40	90002164
165	70104.0852	Blood Monitoring Unit BMU 40	90002165
166	70104.0852	Blood Monitoring Unit BMU 40	90002166
167	70104.0852	Blood Monitoring Unit BMU 40	90002167
168	70104.0852	Blood Monitoring Unit BMU 40	90002168

169	70104.0852	Blood Monitoring Unit BMU 40	90002169
170	70104.0852	Blood Monitoring Unit BMU 40	90002170
171	70104.0852	Blood Monitoring Unit BMU 40	90002171
172	70104.0852	Blood Monitoring Unit BMU 40	90002172
173	70104.0852	Blood Monitoring Unit BMU 40	90002173
174	70104.0852	Blood Monitoring Unit BMU 40	90002174
175	70104.0852	Blood Monitoring Unit BMU 40	90002175
176	70104.0852	Blood Monitoring Unit BMU 40	90002176
177	70104.0852	Blood Monitoring Unit BMU 40	90002177
178	70104.0852	Blood Monitoring Unit BMU 40	90002178
179	70104.0852	Blood Monitoring Unit BMU 40	90002179
180	70104.0852	Blood Monitoring Unit BMU 40	90002180
181	70104.0852	Blood Monitoring Unit BMU 40	90002181
182	70104.0852	Blood Monitoring Unit BMU 40	90002182
183	70104.0852	Blood Monitoring Unit BMU 40	90002183
184	70104.0852	Blood Monitoring Unit BMU 40	90002184
185	70104.0852	Blood Monitoring Unit BMU 40	90002185
186	70104.0852	Blood Monitoring Unit BMU 40	90002186
187	70104.0852	Blood Monitoring Unit BMU 40	90002187
188	70104.0852	Blood Monitoring Unit BMU 40	90002188
189	70104.0852	Blood Monitoring Unit BMU 40	90002189
190	70104.0852	Blood Monitoring Unit BMU 40	90002190
191	70104.0852	Blood Monitoring Unit BMU 40	90002191
192	70104.0852	Blood Monitoring Unit BMU 40	90002192
193	70104.0852	Blood Monitoring Unit BMU 40	90002193
194	70104.0852	Blood Monitoring Unit BMU 40	90002194
195	70104.0852	Blood Monitoring Unit BMU 40	90002195
196	70104.0852	Blood Monitoring Unit BMU 40	90002196
197	70104.0852	Blood Monitoring Unit BMU 40	90002197
198	70104.0852	Blood Monitoring Unit BMU 40	90002198
199	70104.0852	Blood Monitoring Unit BMU 40	90002199
200	70104.0852	Blood Monitoring Unit BMU 40	90002200
201	70104.0852	Blood Monitoring Unit BMU 40	90002201
202	70104.0852	Blood Monitoring Unit BMU 40	90002202
203	70104.0852	Blood Monitoring Unit BMU 40	90002203
204	70104.0852	Blood Monitoring Unit BMU 40	90002204
205	70104.0852	Blood Monitoring Unit BMU 40	90002205
206	70104.0852	Blood Monitoring Unit BMU 40	90002206
207	70104.0852	Blood Monitoring Unit BMU 40	90002207
208	70104.0852	Blood Monitoring Unit BMU 40	90002208
209	70104.0852	Blood Monitoring Unit BMU 40	90002209
210	70104.0852	Blood Monitoring Unit BMU 40	90002210
211	70104.0852	Blood Monitoring Unit BMU 40	90002211
212	70104.0852	Blood Monitoring Unit BMU 40	90002212

213	70104.0852	Blood Monitoring Unit BMU 40	90002213
214	70104.0852	Blood Monitoring Unit BMU 40	90002214
215	70104.0852	Blood Monitoring Unit BMU 40	90002215
216	70104.0852	Blood Monitoring Unit BMU 40	90002216
217	70104.0852	Blood Monitoring Unit BMU 40	90002217
218	70104.0852	Blood Monitoring Unit BMU 40	90002218
219	70104.0852	Blood Monitoring Unit BMU 40	90002219
220	70104.0852	Blood Monitoring Unit BMU 40	90002220
221	70104.0852	Blood Monitoring Unit BMU 40	90002221
222	70104.0852	Blood Monitoring Unit BMU 40	90002222
223	70104.0852	Blood Monitoring Unit BMU 40	90002223
224	70104.0852	Blood Monitoring Unit BMU 40	90002224
225	70104.0852	Blood Monitoring Unit BMU 40	90002225
226	70104.0852	Blood Monitoring Unit BMU 40	90002226
227	70104.0852	Blood Monitoring Unit BMU 40	90002227
228	70104.0852	Blood Monitoring Unit BMU 40	90002228
229	70104.0852	Blood Monitoring Unit BMU 40	90002229
230	70104.0852	Blood Monitoring Unit BMU 40	90002230
231	70104.0852	Blood Monitoring Unit BMU 40	90002231
232	70104.0852	Blood Monitoring Unit BMU 40	90002232
233	70104.0852	Blood Monitoring Unit BMU 40	90002233
234	70104.0852	Blood Monitoring Unit BMU 40	90002234
235	70104.0852	Blood Monitoring Unit BMU 40	90002235
236	70104.0852	Blood Monitoring Unit BMU 40	90002236
237	70104.0852	Blood Monitoring Unit BMU 40	90002237
238	70104.0852	Blood Monitoring Unit BMU 40	90002238
239	70104.0852	Blood Monitoring Unit BMU 40	90002239
240	70104.0852	Blood Monitoring Unit BMU 40	90002240
241	70104.0852	Blood Monitoring Unit BMU 40	90002241
242	70104.0852	Blood Monitoring Unit BMU 40	90002242
243	70104.0852	Blood Monitoring Unit BMU 40	90002243
244	70104.0852	Blood Monitoring Unit BMU 40	90002244
245	70104.0852	Blood Monitoring Unit BMU 40	90002245
246	70104.0852	Blood Monitoring Unit BMU 40	90002246
247	70104.0852	Blood Monitoring Unit BMU 40	90002247
248	70104.0852	Blood Monitoring Unit BMU 40	90002248
249	70104.0852	Blood Monitoring Unit BMU 40	90002249
250	70104.0852	Blood Monitoring Unit BMU 40	90002250
251	70104.0852	Blood Monitoring Unit BMU 40	90002251
252	70104.0852	Blood Monitoring Unit BMU 40	90002252
253	70104.0852	Blood Monitoring Unit BMU 40	90002253
254	70104.0852	Blood Monitoring Unit BMU 40	90002254
255	70104.0852	Blood Monitoring Unit BMU 40	90002255
256	70104.0852	Blood Monitoring Unit BMU 40	90002256

257	70104.0852	Blood Monitoring Unit BMU 40	90002257
258	70104.0852	Blood Monitoring Unit BMU 40	90002258
259	70104.0852	Blood Monitoring Unit BMU 40	90002259
260	70104.0852	Blood Monitoring Unit BMU 40	90002260
261	70104.0852	Blood Monitoring Unit BMU 40	90002261
262	70104.0852	Blood Monitoring Unit BMU 40	90002262
263	70104.0852	Blood Monitoring Unit BMU 40	90002263
264	70104.0852	Blood Monitoring Unit BMU 40	90002264
265	70104.0852	Blood Monitoring Unit BMU 40	90002265
266	70104.0852	Blood Monitoring Unit BMU 40	90002266
267	70104.0852	Blood Monitoring Unit BMU 40	90002267
268	70104.0852	Blood Monitoring Unit BMU 40	90002268
269	70104.0852	Blood Monitoring Unit BMU 40	90002269
270	70104.0852	Blood Monitoring Unit BMU 40	90002270
271	70104.0852	Blood Monitoring Unit BMU 40	90002271
272	70104.0852	Blood Monitoring Unit BMU 40	90002272
273	70104.0852	Blood Monitoring Unit BMU 40	90002273
274	70104.0852	Blood Monitoring Unit BMU 40	90002274
275	70104.0852	Blood Monitoring Unit BMU 40	90002275
276	70104.0852	Blood Monitoring Unit BMU 40	90002276
277	70104.0852	Blood Monitoring Unit BMU 40	90002277
278	70104.0852	Blood Monitoring Unit BMU 40	90002278
279	70104.0852	Blood Monitoring Unit BMU 40	90002279
280	70104.0852	Blood Monitoring Unit BMU 40	90002280
281	70104.0852	Blood Monitoring Unit BMU 40	90002281
282	70104.0852	Blood Monitoring Unit BMU 40	90002282
283	70104.0852	Blood Monitoring Unit BMU 40	90002283
284	70104.0852	Blood Monitoring Unit BMU 40	90002284
285	70104.0852	Blood Monitoring Unit BMU 40	90002285
286	70104.0852	Blood Monitoring Unit BMU 40	90002286
287	70104.0852	Blood Monitoring Unit BMU 40	90002287
288	70104.0852	Blood Monitoring Unit BMU 40	90002288
289	70104.0852	Blood Monitoring Unit BMU 40	90002289
290	70104.0852	Blood Monitoring Unit BMU 40	90002290
291	70104.0852	Blood Monitoring Unit BMU 40	90002291
292	70104.0852	Blood Monitoring Unit BMU 40	90002292
293	70104.0852	Blood Monitoring Unit BMU 40	90002293
294	70104.0852	Blood Monitoring Unit BMU 40	90002294
295	70104.0852	Blood Monitoring Unit BMU 40	90002295
296	70104.0852	Blood Monitoring Unit BMU 40	90002296
297	70104.0852	Blood Monitoring Unit BMU 40	90002297
298	70104.0852	Blood Monitoring Unit BMU 40	90002298
299	70104.0852	Blood Monitoring Unit BMU 40	90002299
300	70104.0852	Blood Monitoring Unit BMU 40	90002300

301	70104.0852	Blood Monitoring Unit BMU 40	90002301
302	70104.0852	Blood Monitoring Unit BMU 40	90002302
303	70104.0852	Blood Monitoring Unit BMU 40	90002303
304	70104.0852	Blood Monitoring Unit BMU 40	90002304
305	70104.0852	Blood Monitoring Unit BMU 40	90002305
306	70104.0852	Blood Monitoring Unit BMU 40	90002306
307	70104.0852	Blood Monitoring Unit BMU 40	90002307
308	70104.0852	Blood Monitoring Unit BMU 40	90002308
309	70104.0852	Blood Monitoring Unit BMU 40	90002309
310	70104.0852	Blood Monitoring Unit BMU 40	90002310
311	70104.0852	Blood Monitoring Unit BMU 40	90002312
312	70104.0852	Blood Monitoring Unit BMU 40	90002313