



URGENT FIELD SAFETY NOTICE

GE Healthcare
4 Hayozma St. PO Box 170
Tirat Ha Carmel, 30200, Israel

December 18, 2022

GEHC Ref. # 40901

To: Director of Clinical/Radiology
Risk Manager/Hospital Administrator
Director of Biomedical Engineering

RE: **GE Healthcare requests STOP USE of certain Nuclear Medicine 600/800 series systems until inspected**

This document contains important information for your product. Please ensure all potential Users in your facility are made aware of this safety notification and actions. Please retain this document for your records.

Safety Issue

For certain Nuclear Medicine systems (see Affected Product Details and Appendix), GE Healthcare has become aware that a mitigation may not be correctly implemented. If this is the case, the detector can fall leading to life-threatening bodily injury.

There have been no injuries reported as a result of this issue.

Actions to be taken by Customer/User


GE Healthcare asks that you **STOP USE** of your Nuclear Medicine system, if your system is listed in the Affected Product Details and Appendix of this letter, until an inspection of your system is completed by GE Healthcare and you have been provided with a letter indicating the system can be returned to clinical use.

Please complete the attached Urgent Medical Device Correction Letter Acknowledgement Response form and send to **Recall.40901@ge.com**

Affected Product Details

The impacted Nuclear Medicine 600/800 Series systems are Brivo 615, Discovery NM 630, Optima 640, Discovery NM/CT 670 DR, Discovery NM/CT 670 ES, Discovery NM/CT 670 Pro, Discovery NM/CT 670 CZT, NM 830, NM 830 ES, NM/CT 850, NM/CT 850 ES, NM/CT 860, NM/CT 860 ES, NM/CT 870 CZT, NM/CT 870 DR, NM/CT 870 ES

Please refer to the Appendix for a list of impacted systems. To locate the serial number of your system, please contact your local Service Representative or find the serial number on the rating plate label which is on the lower left of your Nuclear Medicine gantry:

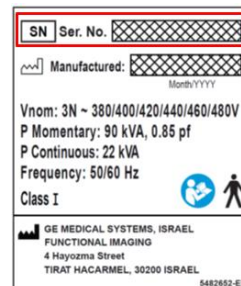

Product Correction

GE Healthcare will correct all affected products at no cost to you. GE Healthcare will inspect your system to ensure that the appropriate mitigation is correctly

implemented to address this potential life-threatening concern. A GE Healthcare representative will contact you to arrange for this correction.

Contact Information If you have any questions or concerns regarding this notification, please contact GE Healthcare Service at 1-800-437-1171 or your local Service Representative.


GE Healthcare confirms that this notice has been notified to the appropriate Regulatory Agency.

Please be assured that maintaining a high level of safety and quality is our highest priority. If you have any questions, please contact us immediately per the contact information above.

Sincerely,



Laila Gurney
Chief Quality & Regulatory Officer
GE Healthcare

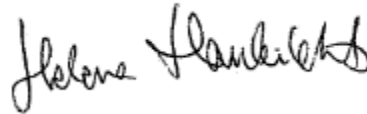

Helena Haukilehto
Medical Director
GE Healthcare



**URGENT MEDICAL DEVICE CORRECTION ACKNOWLEDGEMENT
RESPONSE REQUIRED**

Please complete this form and return it to GE Healthcare promptly upon receipt and no later than 30 days from receipt. This will confirm receipt and understanding of the Urgent Medical Device Correction Letter.

*Customer/Consignee
Name: _____

Street Address: _____

City/State/ZIP/Country: _____

*Customer Email Address: _____

*Customer Phone Number: _____

We acknowledge that we have stopped use of the affected Nuclear Medicine system and will not resume use until an inspection is completed by GE Healthcare. We acknowledge receipt and understanding of the accompanying Urgent Medical Device Correction Letter, and that we have informed appropriate staff and have taken and will take appropriate actions in accordance with that Notification.

Please provide the name of the individual with responsibility who completed this form.

Signature: _____

*Printed Name: _____

*Title: _____

*Date (DD/MM/YYYY): _____

*Indicates Mandatory Fields

Please return completed form by scanning or taking a photo of the completed form and email to: Recall.40901@ge.com



Appendix - Serial Numbers Impacted by FMI Ref 40901

21128	63GX50009	63GX50053	63GY50094	63GZ50135
22404	63GX50010	63GX50054	63GY50095	63GZ50136
26043	63GX50011	63GX50055	63GY50096	63GZ50137
27053	63GX50012	63GX50056	63GY50097	63GZ50138
NT2PG1800001	63GX50013	63GX50057	63GY50098	63GZ50139
NT3AG1900004	63GX50014	63GX50058	63GY50099	63GZ50140
NT3AG1900005	63GX50015	63GX50059	63GY50100	63GZ50141
NT3AG1900015	63GX50016	63GX50060	63GY50101	63GZ50142
NT3AG2000004	63GX50017	63GX50061	63GY50102	63GZ50143
NT3AG2200003	63GX50018	63GX50062	63GY50103	63GZ50144
NT3DG2100005	63GX50019	63GX50063	63GY50104	63GZ50145
NT3DG2100016	63GX50020	63GX50064	63GY50105	63GZ50146
00000589470GE4	63GX50021	63GX50065	63GY50106	63GZ50147
0000063GX50023	63GX50022	63GX50066	63GY50107	63GZ50148
0000063GX50036	63GX50024	63GY50067	63GY50108	63GZ50149
0000063GX50048	63GX50025	63GY50068	63GY50109	63GZ50150
00000860Y62018	63GX50026	63GY50069	63GY50110	63GZ50151
00000860Z62033	63GX50027	63GY50070	63GY50111	63GZ50152
00000870X64014	63GX50028	63GY50071	63GY50112	64GX51001
00000870Y64035	63GX50029	63GY50072	63GY50113	64GY51002
00000870Y64054	63GX50030	63GY50073	63GY50114	64GY51003
00000870Y64060	63GX50031	63GY50074	63GY50115	64GY51004
00000870Z64139	63GX50032	63GY50075	63GY50116	64GY51005
00000870Z64147	63GX50033	63GY50076	63GY50117	64GY51006
00000DRGX56017	63GX50034	63GY50077	63GY50118	64GY51007
00000DRGZ56042	63GX50035	63GY50078	63GY50119	64GY51008
1061000216301/15	63GX50037	63GY50079	63GY50120	64GY51009
1972734	63GX50038	63GY50080	63GY50121	64GY51011
1973098	63GX50039	63GY50081	63GY50122	64GY51012
63GA50153	63GX50040	63GY50082	63GY50123	64GY51013
63GA50154	63GX50041	63GY50083	63GY50124	64GY51014
63GA50155	63GX50042	63GY50084	63GY50125	64GY51015
63GB50164	63GX50043	63GY50085	63GY50126	64GY51016
63GX50001	63GX50044	63GY50086	63GY50127	64GY51017
63GX50002	63GX50045	63GY50087	63GY50128	64GY51018
63GX50003	63GX50046	63GY50088	63GY50129	64GY51019
63GX50004	63GX50047	63GY50089	63GY50130	64GY51020
63GX50005	63GX50049	63GY50090	63GY50131	64GY51021
63GX50006	63GX50050	63GY50091	63GZ50132	64GY51022
63GX50007	63GX50051	63GY50092	63GZ50133	64GY51023
63GX50008	63GX50052	63GY50093	63GZ50134	64GY51024

64GY51025	830A60305	830X60025	830Y60067	830Y60108
64GZ51026	830A60306	830X60026	830Y60068	830Y60109
64GZ51027	830A60307	830X60027	830Y60069	830Y60110
64GZ51028	830A60308	830X60028	830Y60070	830Y60111
64GZ51029	830A60309	830Y60029	830Y60071	830Y60112
64GZ51030	830A60310	830Y60030	830Y60072	830Y60113
64GZ51031	830A60311	830Y60032	830Y60073	830Y60114
830A60271	830A60334	830Y60033	830Y60074	830Y60115
830A60272	A60343830	830Y60034	830Y60075	830Y60116
830A60273	B60393830	830Y60035	830Y60076	830Y60117
830A60274	B60417830	830Y60036	830Y60077	830Y60118
830A60275	B60420830	830Y60037	830Y60078	830Y60119
830A60276	B60437830	830Y60038	830Y60079	830Y60120
830A60277	B60438830	830Y60039	830Y60080	830Y60121
830A60278	830B60455	830Y60040	830Y60081	830Y60122
830A60279	830B60456	830Y60041	830Y60082	830Y60123
830A60280	830B60457	830Y60042	830Y60083	830Y60124
830A60281	830X60001	830Y60043	830Y60084	830Y60125
830A60282	830X60002	830Y60044	830Y60085	830Y60126
830A60283	830X60003	830Y60045	830Y60086	830Y60127
830A60284	830X60004	830Y60046	830Y60087	830Y60128
830A60285	830X60005	830Y60047	830Y60088	830Y60129
830A60286	830X60006	830Y60048	830Y60089	830Y60130
830A60287	830X60007	830Y60049	830Y60090	830Y60131
830A60288	830X60008	830Y60050	830Y60091	830Y60132
830A60289	830X60009	830Y60051	830Y60092	830Y60133
830A60290	830X60010	830Y60052	830Y60093	830Y60134
830A60291	830X60011	830Y60053	830Y60094	830Y60135
830A60292	830X60012	830Y60054	830Y60095	830Y60136
830A60293	830X60013	830Y60055	830Y60096	830Y60137
830A60294	830X60014	830Y60056	830Y60097	830Y60138
830A60295	830X60015	830Y60057	830Y60098	830Y60140
830A60296	830X60016	830Y60058	830Y60099	830Y60141
830A60297	830X60017	830Y60059	830Y60100	830Y60142
830A60298	830X60018	830Y60060	830Y60101	830Y60143
830A60299	830X60019	830Y60061	830Y60102	830Y60144
830A60300	830X60020	830Y60062	830Y60103	830Y60145
830A60301	830X60021	830Y60063	830Y60104	830Y60146
830A60302	830X60022	830Y60064	830Y60105	830Y60147
830A60303	830X60023	830Y60065	830Y60106	830Z60148
830A60304	830X60024	830Y60066	830Y60107	830Z60149

830Z60150	830Z60191	830Z60232	83EA88008	850Y61016
830Z60151	830Z60192	830Z60233	83EA88009	850Y61017
830Z60152	830Z60193	830Z60234	83EA88010	850Y61018
830Z60153	830Z60194	830Z60235	EB8802083	850Y61019
830Z60154	830Z60195	830Z60236	83EB88021	850Y61020
830Z60155	830Z60196	830Z60237	83EZ88001	850Y61021
830Z60156	830Z60197	830Z60238	83EZ88002	850Y61022
830Z60157	830Z60198	830Z60239	83EZ88003	850Y61023
830Z60158	830Z60199	830Z60240	83EZ88004	850Y61024
830Z60159	830Z60200	830Z60241	83EZ88005	850Y61025
830Z60160	830Z60201	830Z60242	850A61062	850Y61026
830Z60161	830Z60202	830Z60243	850A61063	850Z61027
830Z60162	830Z60203	830Z60244	850A61064	850Z61029
830Z60163	830Z60204	830Z60245	850A61065	850Z61030
830Z60164	830Z60205	830Z60246	850A61066	850Z61031
830Z60165	830Z60206	830Z60247	850A61067	850Z61032
830Z60166	830Z60207	830Z60248	850A61068	850Z61033
830Z60167	830Z60208	830Z60249	850A61069	850Z61034
830Z60168	830Z60209	830Z60250	850A61070	850Z61035
830Z60169	830Z60210	830Z60251	850A61071	850Z61036
830Z60170	830Z60211	830Z60252	850A61072	850Z61037
830Z60171	830Z60212	830Z60253	850A61073	850Z61038
830Z60172	830Z60213	830Z60254	850A61074	850Z61039
830Z60173	830Z60214	830Z60255	850A61075	850Z61040
830Z60174	830Z60215	830Z60256	850A61076	850Z61041
830Z60175	830Z60216	830Z60257	B61100850	850Z61042
830Z60176	830Z60217	830Z60258	B61103850	850Z61043
830Z60177	830Z60218	830Z60259	850X61001	850Z61044
830Z60178	830Z60219	830Z60260	850X61002	850Z61045
830Z60179	830Z60220	830Z60261	850X61004	850Z61046
830Z60180	830Z60221	830Z60262	850X61005	850Z61047
830Z60181	830Z60222	830Z60263	850Y61006	850Z61048
830Z60182	830Z60223	830Z60264	850Y61007	850Z61049
830Z60183	830Z60224	830Z60265	850Y61008	850Z61051
830Z60184	830Z60225	830Z60266	850Y61009	850Z61052
830Z60185	830Z60226	830Z60267	850Y61010	850Z61053
830Z60186	830Z60227	830Z60268	850Y61011	850Z61054
830Z60187	830Z60228	830Z60269	850Y61012	850Z61055
830Z60188	830Z60229	830Z60270	850Y61013	850Z61056
830Z60189	830Z60230	83EA88006	850Y61014	850Z61057
830Z60190	830Z60231	83EA88007	850Y61015	850Z61058

850Z61059	85EY67036	860A62063	860Z62041	B64245870
850Z61060	85EY67037	860A62064	860Z62042	B64274870
850Z61061	85EY67038	860A62065	860Z62043	B64288870
85EA67063	85EY67039	B62102860	860Z62044	B64294870
85EA67064	85EY67040	B62118860	860Z62045	B64304870
85EA67065	85EY67041	860X62001	860Z62046	870X64001
85EA67066	85EY67042	860X62002	860Z62047	870X64002
85EX67001	85EY67043	860X62003	86EA76002	870X64003
85EX67002	85EY67044	860Y62004	86EA76003	870X64004
85EX67003	85EY67045	860Y62005	86EA76004	870X64005
85EX67004	85EY67046	860Y62006	86EZ76001	870X64007
85EX67005	85EY67047	860Y62007	870A64152	870X64008
85EX67006	85EY67048	860Y62009	870A64153	870X64009
85EX67007	85EY67049	860Y62010	870A64154	870X64010
85EX67008	85EY67050	860Y62011	870A64155	870X64011
85EX67009	85EZ67051	860Y62012	870A64156	870X64012
85EX67010	85EZ67052	860Y62013	870A64157	870X64013
85EX67011	85EZ67053	860Y62014	870A64158	870X64015
85EX67012	85EZ67054	860Y62015	870A64159	870X64016
85EX67013	85EZ67055	860Y62016	870A64160	870X64017
85EX67014	85EZ67056	860Y62017	870A64161	870X64019
85EX67016	85EZ67057	860Y62019	870A64162	870X64020
85EX67017	85EZ67058	860Y62020	870A64163	870X64021
85EX67018	85EZ67059	860Y62021	870A64164	870X64022
85EX67019	85EZ67060	860Y62022	870A64165	870X64023
85EX67020	85EZ67061	860Y62023	870A64166	870X64024
85EX67021	85EZ67062	860Y62024	870A64168	870X64025
85EX67022	860A62048	860Y62025	870A64169	870X64026
85EX67023	860A62049	860Z62026	870A64170	870X64027
85EX67024	860A62050	860Z62027	870A64171	870X64028
85EY67015	860A62052	860Z62028	870A64172	870X64029
85EY67025	860A62053	860Z62030	870A64173	870X64030
85EY67026	860A62054	860Z62031	870A64174	870X64031
85EY67027	860A62055	860Z62032	870A64175	870X64032
85EY67028	860A62056	860Z62034	870A64176	870Y64033
85EY67030	860A62057	860Z62035	870A64177	870Y64034
85EY67031	860A62058	860Z62036	870A64179	870Y64036
85EY67032	860A62059	860Z62037	870A64180	870Y64037
85EY67033	860A62060	860Z62038	A64186870	870Y64038
85EY67034	860A62061	860Z62039	870A64201	870Y64039
85EY67035	860A62062	860Z62040	870B64224	870Y64040

870Y64041	870Y64084	870Z64127	87CX63014	87EA77011
870Y64042	870Y64085	870Z64128	87CX63015	87EZ77001
870Y64043	870Y64086	870Z64129	87CY63016	87EZ77002
870Y64044	870Y64087	870Z64130	87CY63017	87EZ77003
870Y64045	870Y64088	870Z64131	87CY63018	87EZ77004
870Y64046	870Y64089	870Z64132	87CY63019	87EZ77005
870Y64047	870Y64090	870Z64133	87CY63020	87EZ77006
870Y64048	870Y64091	870Z64134	87CY63021	87EZ77007
870Y64049	870Y64092	870Z64135	87CY63022	87EZ77008
870Y64050	870Y64094	870Z64136	87CY63023	8BBB30001
870Y64051	870Y64095	870Z64137	87CY63024	8BBB30002
870Y64052	870Y64096	870Z64138	87CY63025	8BCB40001
870Y64053	870Y64097	870Z64140	87CY63026	CZGA55003
870Y64055	870Z64098	870Z64141	87CY63027	CZGA55004
870Y64056	870Z64099	870Z64142	87CY63028	CZGY55001
870Y64057	870Z64101	870Z64143	87CY63029	CZGZ55002
870Y64058	870Z64102	870Z64144	87CY63030	DRGX56001
870Y64059	870Z64103	870Z64145	87CY63031	DRGX56002
870Y64061	870Z64104	870Z64146	87CY63032	DRGX56003
870Y64062	870Z64105	870Z64148	87CY63033	DRGX56004
870Y64063	870Z64106	870Z64149	87CY63034	DRGX56005
870Y64064	870Z64107	870Z64150	87CY63035	DRGX56006
870Y64065	870Z64108	870Z64151	87CY63036	DRGX56007
870Y64066	870Z64109	87CA63053	87CY63037	DRGX56008
870Y64067	870Z64110	87CA63054	87CY63038	DRGX56009
870Y64068	870Z64111	87CA63055	87CZ63039	DRGX56010
870Y64069	870Z64112	87CA83001	87CZ63040	DRGX56011
870Y64070	870Z64113	87CA83004	87CZ63041	DRGX56012
870Y64071	870Z64114	87CX63001	87CZ63042	DRGX56014
870Y64072	870Z64115	87CX63002	87CZ63043	DRGX56015
870Y64073	870Z64116	87CX63003	87CZ63044	DRGX56016
870Y64074	870Z64117	87CX63004	87CZ63045	DRGX56018
870Y64075	870Z64118	87CX63005	87CZ63046	DRGX56019
870Y64076	870Z64119	87CX63006	87CZ63047	DRGX56020
870Y64077	870Z64120	87CX63007	87CZ63048	DRGX56021
870Y64078	870Z64121	87CX63008	87CZ63049	DRGX56022
870Y64079	870Z64122	87CX63009	87CZ63050	DRGX56023
870Y64080	870Z64123	87CX63010	87CZ63051	DRGY56024
870Y64081	870Z64124	87CX63011	87CZ63052	DRGY56026
870Y64082	870Z64125	87CX63012	87EA77009	DRGY56027
870Y64083	870Z64126	87CX63013	87EA77010	DRGY56028

DRGY56029	NT2PG1900005	NT3AG1900018	NT3AG2200010	NT3CG2200002
DRGY56030	NT2PG1900006	NT3AG2000001	NT3AG2200011	NT3CG2200003
DRGY56032	NT2PG1900007	NT3AG2000002	NT3AG2200012	NT3CG2200004
DRGY56033	NT2PG1900008	NT3AG2000003	NT3AG2200013	NT3DG1800001
DRGY56034	NT2PG1900009	NT3AG2000005	NT3AG2200014BH	NT3DG1800002BH
DRGY56035	NT2PG1900010	NT3AG2000005BH	NT3AG2200015	NT3DG1800003BH
DRGY56036	NT2PG1900011	NT3AG2000006	NT3AG2200016	NT3DG1900001
DRGY56037	NT2PG1900012	NT3AG2000008	NT3BG1900001BH	NT3DG1900002
DRGY56038	NT2PG2000001BH	NT3AG2000009	NT3BG1900002	NT3DG1900003
DRGZ56039	NT2QG1800001	NT3AG2000010	NT3BG1900003BH	NT3DG1900004
DRGZ56040	NT2QG1900001	NT3AG2000011	NT3BG2000001	NT3DG1900004BH
DRGZ56041	NT2QG1900002	NT3AG2000012	NT3BG2000002	NT3DG1900005
DRGZ56042	NT2QG1900003	NT3AG2000013	NT3BG2000003	NT3DG1900005BH
DRGZ56043	NT2QG2000001	NT3AG2000015	NT3BG2000004	NT3DG1900006BH
DRGZ56044	NT2QG2000002BH	NT3AG2000016	NT3BG2100001	NT3DG1900007BH
DRGZ56045	NT2QG2000003	NT3AG2000017	NT3BG2100003	NT3DG1900009
DRGZ56046	NT2RG1800002	NT3AG2100001	NT3BG2100004	NT3DG1900009BH
DRGZ56047	NT2RG1900001	NT3AG2100001BH	NT3BG2100005	NT3DG1900010
DRGZ56048	NT2RG1900002	NT3AG2100002	NT3BG2200001	NT3DG1900011
DRGZ56049	NT2RG1900003	NT3AG2100003	NT3BG2200002	NT3DG1900011BH
DRGZ56050	NT2RG1900004BH	NT3AG2100004BH	NT3BG2200005	NT3DG1900012
DRGZ56051	NT2RG1900005	NT3AG2100005	NT3BG2200006	NT3DG1900013
DRGZ56052	NT2RG2000001	NT3AG2100006BH	NT3CG1900002	NT3DG1900014
DRGZ56053	NT2RG2200001	NT3AG2100007	NT3CG1900003	NT3DG1900015
DRGZ56054	NT2UG2100001	NT3AG2100008	NT3CG1900005	NT3DG2000001
DRGZ56055	NT2UG2100002	NT3AG2100009	NT3CG1900006	NT3DG2000002
DRGZ56056	NT3AG1800001	NT3AG2100010	NT3CG1900007	NT3DG2000003
ESGA53007	NT3AG1800002	NT3AG2100011	NT3CG1900008	NT3DG2000004
ESGY53001	NT3AG1900001	NT3AG2100012	NT3CG1900009	NT3DG2000005
ESGY53002	NT3AG1900003	NT3AG2100013	NT3CG1900010	NT3DG2000006
ESGY53003	NT3AG1900006	NT3AG2100014	NT3CG2000001BH	NT3DG2000008
ESGZ53004	NT3AG1900007BH	NT3AG2100015	NT3CG2000002BH	NT3DG2000010
ESGZ53005	NT3AG1900008	NT3AG2100016	NT3CG2000003	NT3DG2000011
ESGZ53006	NT3AG1900009	NT3AG2200001	NT3CG2000004	NT3DG2000012
ESPX54001	NT3AG1900010	NT3AG2200002BH	NT3CG2100001	NT3DG2000014BH
ESPY54002	NT3AG1900011	NT3AG2200004	NT3CG2100002	NT3DG2000015
ESPY54003	NT3AG1900012	NT3AG2200005	NT3CG2100003	NT3DG2000016
NT2PG1900001	NT3AG1900013	NT3AG2200006	NT3CG2100004	NT3DG2100001
NT2PG1900002	NT3AG1900014	NT3AG2200007BH	NT3CG2100005	NT3DG2100002BH
NT2PG1900003	NT3AG1900016	NT3AG2200008	NT3CG2100006	NT3DG2100003
NT2PG1900004	NT3AG1900017	NT3AG2200009	NT3CG2200001	NT3DG2100004

NT3DG2100006	PRGA52069	PRGZ52041
NT3DG2100007	PRGA52070	PRGZ52042
NT3DG2100008	PRGX52001	PRGZ52043
NT3DG2100009	PRGX52002	PRGZ52044
NT3DG2100010	PRGX52003	PRGZ52045
NT3DG2100011	PRGX52004	PRGZ52046
NT3DG2100012	PRGX52005	PRGZ52047
NT3DG2100013	PRGX52006	PRGZ52048
NT3DG2100014	PRGX52007	PRGZ52049
NT3DG2100015	PRGX52008	PRGZ52050
NT3DG2100018	PRGX52009	PRGZ52051
NT3DG2200001	PRGX52010	PRGZ52052
NT3DG2200002	PRGY52011	PRGZ52053
NT3DG2200003	PRGY52012	PRGZ52054
NT3DG2200004	PRGY52013	PRGZ52055
NT3DG2200004BH	PRGY52014	PRGZ52056
NT3DG2200005	PRGY52015	PRGZ52057
NT3DG2200006	PRGY52016	PRGZ52058
NT3DG2200007	PRGY52017	PRGZ52059
NT3DG2200008	PRGY52018	
NT3DG2200009	PRGY52020	
NT3DG2200010	PRGY52021	
NT3DG2200011	PRGY52022	
NT3DG2200012	PRGY52023	
NT3DG2200013	PRGY52024	
NT3DG2200014	PRGY52025	
NT3DG2200015	PRGY52026	
NT3DG2200016	PRGY52027	
NT3DG2200017	PRGY52028	
NT3DG2200018	PRGY52029	
NT3DG2200019	PRGY52030	
ONMX26515	PRGY52031	
PRGA52060	PRGY52032	
PRGA52061	PRGY52033	
PRGA52062	PRGY52034	
PRGA52063	PRGY52035	
PRGA52064	PRGY52036	
PRGA52065	PRGY52037	
PRGA52066	PRGY52038	
PRGA52067	PRGY52039	
PRGA52068	PRGZ52040	