Avis De Sécurité / Field Safety Notice**FSCA-03-2023-1****Urology Suction Tubing with Pump Body**

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*This letter contains important information which require your **immediate attention**.*

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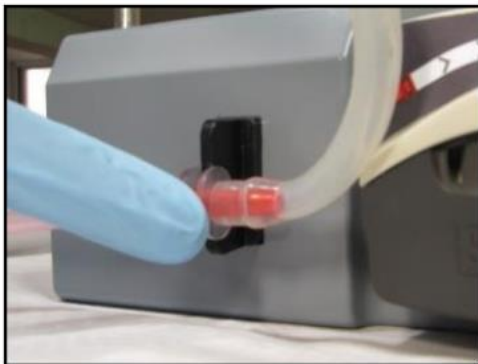
Dear Customer,

*ROCAMED France manufacturer of devices distributed in **Appendix 1 and 2**, are conducting a safety corrective action regarding lots of surgical irrigation/ aspiration tubing set.*

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
ROCAMED SAM has received complaints about Urology Suction Tubing with Pump Body of reference SOUR9200ST12 of lot R006497.

The fault is on the suction pipe. The part that fits in the suction pump head is missing as well as the poka yoke allowing the tubing to be inserted in the right direction.

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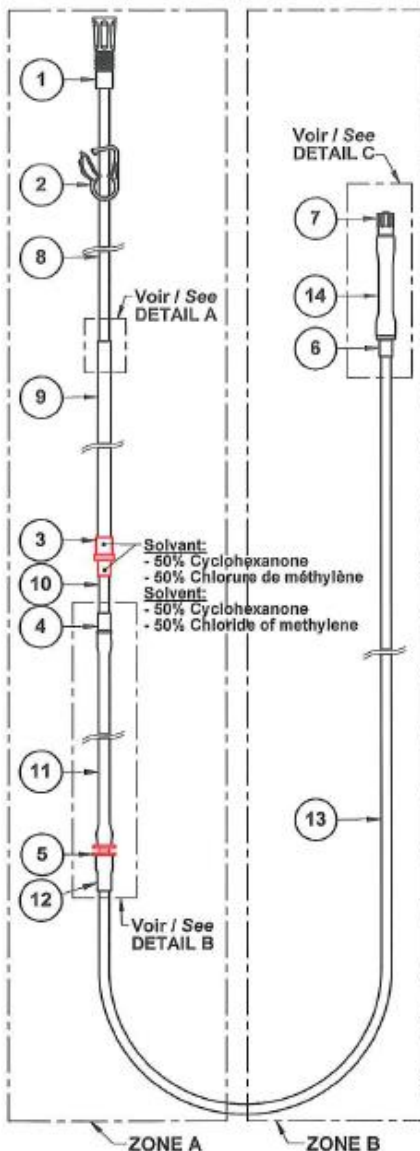The missing components of reference ONG499206V2 are:

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After a review of the batch files and investigations, we were able to confirm a drift in the production process on this batch R006497.

The product specification for reference SOUR9200ST12 uses ONG499206V2 (Suction Urology tubing with pump body), while the production site used ONG499206 (without pump body).

However, as a preventive measure, this FSN will be shared with all customers who have received the impacted batch R006497 mentioned in appendix 1 and 2.

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Any establishment having in stock the batches listed in appendix 1 and 2, are requested to return them to us.

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Please then contact your usual contact to arrange for the turn.

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If you have any question about this Field Safety Notice, please contact us, at telephone number +377 97 98 41 93 or email mr@promempla.com.

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ANSM, Swissmedic, SAHPRA, Ministry of Health, FAMHP - Federal Agency for Medicines and Health Products and Market Surveillance & Vigilance, Dutch Health and Youth Care Inspectorate (IGJ).

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No risk to patient safety:

- *Either the end user has a vacuum suction system and the tubing remains usable.*
- *Either the end user has no other system and the tubing is unusable.*
 - ⇒ *Need to take another set of irrigation/suction tubing but from another batch.*
 - ⇒ *Increase in anesthesia time if another set from another batch.*
 - ⇒ *Cancellation of the intervention if no set from another batch.*

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In issuing this FSCA, we are aware that we have taken a conservative approach to the issue and we are counting on your understanding and full support to ensure the prompt and effective follow-up of our security action.

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We apologize for any inconvenience this may cause you and your customers and remain at your disposal for any other recommendations or requests.

Cordialement
Yours faithfully