

Urgent Field Safety Notice

Commercial name/Model: *BeneHeart D1 Automated External Defibrillator*

FSCA-identifier: *CP2404-JH04699*

Type of action: *Manual upgrade*

27th May. 2024

Attention:

Dear Sir or Madam,

Through the continuous monitoring of the products distributed by Mindray, we have become aware of a potential issue may be associated with BeneHeart D1 Automated External Defibrillator. This letter is intended to provide you with information as following:

Details on affected devices:

The affected serial numbers (SN) of the BeneHeart D1 can be identified from the label. The serial numbers of affected devices are listed in ***appendix 1***

Description of the problem:

Mindray has identified a potential issue associated with BeneHeart D1 Automated External Defibrillator. It was found that, BeneHeart D1 can support MR60, MR61, MR62, MR63 electrode pads and some customers may purchase more than one model of electrode pads. When there are different models of electrode pads connected to the D1, different prompts for the connection of pads appear on the device. However, those different connection prompts might be confusing for the customers.

To date, Mindray has not received any report of serious incidents such as death or serious injury due to the above issue. A new version of BeneHeart D1 product manual has been released, which adds electrode pad connection differences and a more detailed AED procedure to optimise the potential issue mentioned above.

Advice on action to be taken by the Hospital administrator:

1. Please pass the FSN to anyone in your organization who needs to know this information, and to organizations or individuals to whom the affected products may have been sold or transferred
2. You can continue to use the device when you receive this field safety notice (FSN) and the updated user manual.
3. Our local Mindray Service Representatives or cooperative distributors will contact you as soon as possible to provide the new version of the product manuals of the affected units.

Advice on action to be taken by the distributor:

1. Please pass the FSN to anyone in your facility who needs to know this information, and to organizations or individuals to whom the affected products may have been distributed.
2. If any BeneHeart D1 Automated External Defibrillators in your facility is on the affected list, please sell these devices to customers with the updated manual. Mindray's Service Representatives will contact you to provide the new version of the product manual of

the affected units.

Transmission of this Field Safety Notice:

This Notice needs to be passed on all those who need to be aware within your organization or to any organization where the potentially affected device(s) have been transferred.

Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action.

We would be grateful if you could confirm receipt of this letter. Please fill in below Acknowledgement Form and return via E-mail or Fax.

Contact reference person:

We apologize for the inconvenience caused by this situation. If you have any questions, please contact with your local Mindray Customer Service Engineer or designated Technical Support Engineer.

Organization: SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.

E-Mail: hilfe@mindray.de

This Notice has been notified the appropriate Regulatory Agency.

Signature:



Matthew Ding

Representative of Post-market Surveillance of Quality Center

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.
Mindray Building, Keji 12th Road South, High-tech Industrial Park,
Nanshan, Shenzhen 518057, P.R. China
Tel: 0086 755 8188 5638
Email : mr@mindray.com

Acknowledgement Form

=====
Confirmation of Receipt of Field Safety Notice
=====

Commercial name/Model: *BeneHeart D1 Automated External Defibrillator*

FSCA-identifier: *CP2404-JH04699*

Type of FSCA : *Manual upgrade*
=====

Please fill in this form and return this confirmation by E-mail immediately.

Email: hilfe@mindray.de

Name: _____

Tel. No.: _____

E-mail address: _____

Date and Signature: _____

Address of the Organization:

Appendix 1 List of Affected Devices

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
1	BeneHeart D1	FQ-02047645	41	BeneHeart D1	FQ-1C061927
2	BeneHeart D1	FQ-03047735	42	BeneHeart D1	FQ-23062821
3	BeneHeart D1	FQ-03047736	43	BeneHeart D1	FQ-23062823
4	BeneHeart D1	FQ-03047737	44	BeneHeart D1	FQ-24062862
5	BeneHeart D1	FQ-06050065	45	BeneHeart D1	FQ-24062863
6	BeneHeart D1	FQ-06050066	46	BeneHeart D1	FQ-26063251
7	BeneHeart D1	FQ-06050067	47	BeneHeart D1	FQ-26063252
8	BeneHeart D1	FQ-08051632	48	BeneHeart D1	FQ-26063253
9	BeneHeart D1	FQ-0A053881	49	BeneHeart D1	FQ-27063442
10	BeneHeart D1	FQ-0A053882	50	BeneHeart D1	FQ-28063565
11	BeneHeart D1	FQ-0A053883	51	BeneHeart D1	FQ-28063568
12	BeneHeart D1	FQ-0A053993	52	BeneHeart D1	FQ-28063577
13	BeneHeart D1	FQ-0C055347	53	BeneHeart D1	FQ-2A063747
14	BeneHeart D1	FQ-0C055348	54	BeneHeart D1	FQ-2B063842
15	BeneHeart D1	FQ-0C055352	55	BeneHeart D1	FQ-2B063843
16	BeneHeart D1	FQ-13057324	56	BeneHeart D1	FQ-2B063844
17	BeneHeart D1	FQ-13057492	57	BeneHeart D1	FQ-2B063845
18	BeneHeart D1	FQ-14058413	58	BeneHeart D1	FQ-2B063846
19	BeneHeart D1	FQ-14058414	59	BeneHeart D1	FQ-2C064113
20	BeneHeart D1	FQ-14058471	60	BeneHeart D1	FQ-2C064137
21	BeneHeart D1	FQ-16059068	61	BeneHeart D1	FQ-32064257
22	BeneHeart D1	FQ-16059170	62	BeneHeart D1	FQ-34064291
23	BeneHeart D1	FQ-16059171	63	BeneHeart D1	FQ-34064292
24	BeneHeart D1	FQ-16059172	64	BeneHeart D1	FQ-36064342
25	BeneHeart D1	FQ-17059504	65	BeneHeart D1	FQ-36064343
26	BeneHeart D1	FQ-17059505	66	BeneHeart D1	FQ-36064453
27	BeneHeart D1	FQ-17059506	67	BeneHeart D1	FQ-37064467
28	BeneHeart D1	FQ-17059507	68	BeneHeart D1	FQ-37064468
29	BeneHeart D1	FQ-17060118	69	BeneHeart D1	FQ-38064481
30	BeneHeart D1	FQ-17060119	70	BeneHeart D1	FQ-38064567
31	BeneHeart D1	FQ-19061573	71	BeneHeart D1	FQ-38064568
32	BeneHeart D1	FQ-1A061585	72	BeneHeart D1	FQ-38064569
33	BeneHeart D1	FQ-1A061586	73	BeneHeart D1	FQ-38064575
34	BeneHeart D1	FQ-1A061616	74	BeneHeart D1	FQ-39064576
35	BeneHeart D1	FQ-1B061622	75	BeneHeart D1	FQ-39064577
36	BeneHeart D1	FQ-1B061624	76	BeneHeart D1	FQ-39064578
37	BeneHeart D1	FQ-1C061923	77	BeneHeart D1	FQ-39064579
38	BeneHeart D1	FQ-1C061924	78	BeneHeart D1	FQ-3A000080
39	BeneHeart D1	FQ-1C061925	79	BeneHeart D1	FQ-3A000081
40	BeneHeart D1	FQ-1C061926	80	BeneHeart D1	FQ-3A000083

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
81	BeneHeart D1	FQ-3A000084	121	BeneHeart D1	FQ-3C000282
82	BeneHeart D1	FQ-3A000085	122	BeneHeart D1	FQ-3C000283
83	BeneHeart D1	FQ-3A000086	123	BeneHeart D1	FQ-3C000284
84	BeneHeart D1	FQ-3A000087	124	BeneHeart D1	FQ-3C000285
85	BeneHeart D1	FQ-3A064741	125	BeneHeart D1	FQ-42000376
86	BeneHeart D1	FQ-3B000115	126	BeneHeart D1	FQ-42000377
87	BeneHeart D1	FQ-3B000128	127	BeneHeart D1	FQ-42000395
88	BeneHeart D1	FQ-3B000129	128	BeneHeart D1	FQ-42064987
89	BeneHeart D1	FQ-3B000185	129	BeneHeart D1	FQ-42064988
90	BeneHeart D1	FQ-3B000186	130	BeneHeart D1	FQ-46000655
91	BeneHeart D1	FQ-3B064749	131	BeneHeart D1	FQ-46000656
92	BeneHeart D1	FQ-3B064750	132	BeneHeart D1	FQ-46000660
93	BeneHeart D1	FQ-3B064755	133	BeneHeart D1	FQ-46000664
94	BeneHeart D1	FQ-3C000188	134	BeneHeart D1	FQ-46000665
95	BeneHeart D1	FQ-3C000189	135	BeneHeart D1	FQ-46000668
96	BeneHeart D1	FQ-3C000190	136	BeneHeart D1	FQ-46000669
97	BeneHeart D1	FQ-3C000191	137	BeneHeart D1	FQ-46000670
98	BeneHeart D1	FQ-3C000192	138	BeneHeart D1	FQ-46000671
99	BeneHeart D1	FQ-3C000193	139	BeneHeart D1	FQ-46000672
100	BeneHeart D1	FQ-3C000194	140	BeneHeart D1	FQ-46000673
101	BeneHeart D1	FQ-3C000195	141	BeneHeart D1	FQ-46000675
102	BeneHeart D1	FQ-3C000196	142	BeneHeart D1	FQ-46000676
103	BeneHeart D1	FQ-3C000197	143	BeneHeart D1	FQ-46000677
104	BeneHeart D1	FQ-3C000198	144	BeneHeart D1	FQ-46000678
105	BeneHeart D1	FQ-3C000199	145	BeneHeart D1	FQ-46000679
106	BeneHeart D1	FQ-3C000200	146	BeneHeart D1	FQ-46000680
107	BeneHeart D1	FQ-3C000201	147	BeneHeart D1	FQ-46000717
108	BeneHeart D1	FQ-3C000202	148	BeneHeart D1	FQ-46000935
109	BeneHeart D1	FQ-3C000204	149	BeneHeart D1	FQ-46000936
110	BeneHeart D1	FQ-3C000205	150	BeneHeart D1	FQ-46000937
111	BeneHeart D1	FQ-3C000206	151	BeneHeart D1	FQ-46000938
112	BeneHeart D1	FQ-3C000208	152	BeneHeart D1	FQ-46000939
113	BeneHeart D1	FQ-3C000274	153	BeneHeart D1	FQ-46000952T
114	BeneHeart D1	FQ-3C000275	154	BeneHeart D1	FQ-46000954T
115	BeneHeart D1	FQ-3C000276	155	BeneHeart D1	FQ-46000955T
116	BeneHeart D1	FQ-3C000277	156	BeneHeart D1	FQ-46000956T
117	BeneHeart D1	FQ-3C000278	157	BeneHeart D1	FQ-46000957T
118	BeneHeart D1	FQ-3C000279	158	BeneHeart D1	FQ-46000958T
119	BeneHeart D1	FQ-3C000280	159	BeneHeart D1	FQ-47000998
120	BeneHeart D1	FQ-3C000281	160	BeneHeart D1	FQ-47000999

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
161	BeneHeart D1	FQ-47001000	201	BeneHeart D1	FQ-5A003542
162	BeneHeart D1	FQ-47001001	202	BeneHeart D1	FQ-5B003814
163	BeneHeart D1	FQ-47001254T	203	BeneHeart D1	FQ-5B003865
164	BeneHeart D1	FQ-47001255T	204	BeneHeart D1	FQ-5B003866
165	BeneHeart D1	FQ-47001256T	205	BeneHeart D1	FQ-5C004097
166	BeneHeart D1	FQ-47001257T	206	BeneHeart D1	FQ-61004209
167	BeneHeart D1	FQ-48001264T	207	BeneHeart D1	FQ-61004212
168	BeneHeart D1	FQ-49001378	208	BeneHeart D1	FQ-62004422
169	BeneHeart D1	FQ-49001379	209	BeneHeart D1	FQ-62004527
170	BeneHeart D1	FQ-49001380	210	BeneHeart D1	FQ-63004586
171	BeneHeart D1	FQ-49001381	211	BeneHeart D1	FQ-63004676
172	BeneHeart D1	FQ-49001382	212	BeneHeart D1	FQ-63004682
173	BeneHeart D1	FQ-49001383	213	BeneHeart D1	FQ-63004683
174	BeneHeart D1	FQ-49001384	214	BeneHeart D1	FQ-63004684
175	BeneHeart D1	FQ-49001385	215	BeneHeart D1	FQ-63004685
176	BeneHeart D1	FQ-49001386	216	BeneHeart D1	FQ-63004686
177	BeneHeart D1	FQ-49001387	217	BeneHeart D1	FQ-63004687
178	BeneHeart D1	FQ-4C002100	218	BeneHeart D1	FQ-63004688
179	BeneHeart D1	FQ-4C002152	219	BeneHeart D1	FQ-63004689
180	BeneHeart D1	FQ-4C002154	220	BeneHeart D1	FQ-63004755
181	BeneHeart D1	FQ-4C002156	221	BeneHeart D1	FQ-63004836
182	BeneHeart D1	FQ-4C002158	222	BeneHeart D1	FQ-63004860
183	BeneHeart D1	FQ-4C002159	223	BeneHeart D1	FQ-63004861
184	BeneHeart D1	FQ-4C002161	224	BeneHeart D1	FQ-63004862
185	BeneHeart D1	FQ-4C002164	225	BeneHeart D1	FQ-63004883
186	BeneHeart D1	FQ-4C002165	226	BeneHeart D1	FQ-64004932
187	BeneHeart D1	FQ-4C002170	227	BeneHeart D1	FQ-64004933
188	BeneHeart D1	FQ-4C002173	228	BeneHeart D1	FQ-64004934
189	BeneHeart D1	FQ-4C002176	229	BeneHeart D1	FQ-64004935
190	BeneHeart D1	FQ-53002391	230	BeneHeart D1	FQ-64004936
191	BeneHeart D1	FQ-53002392	231	BeneHeart D1	FQ-64004937
192	BeneHeart D1	FQ-53002393	232	BeneHeart D1	FQ-64004938
193	BeneHeart D1	FQ-59003212	233	BeneHeart D1	FQ-64004939
194	BeneHeart D1	FQ-59003213	234	BeneHeart D1	FQ-64004940
195	BeneHeart D1	FQ-59003214	235	BeneHeart D1	FQ-64004941
196	BeneHeart D1	FQ-5A003376	236	BeneHeart D1	FQ-64004942
197	BeneHeart D1	FQ-5A003377	237	BeneHeart D1	FQ-64004943
198	BeneHeart D1	FQ-5A003378	238	BeneHeart D1	FQ-64004944
199	BeneHeart D1	FQ-5A003380	239	BeneHeart D1	FQ-64004945
200	BeneHeart D1	FQ-5A003381	240	BeneHeart D1	FQ-64005069

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
241	BeneHeart D1	FQ-64005070	281	BeneHeart D1	FQ-66005997
242	BeneHeart D1	FQ-64005071	282	BeneHeart D1	FQ-66005998
243	BeneHeart D1	FQ-64005072	283	BeneHeart D1	FQ-66005999
244	BeneHeart D1	FQ-64005073	284	BeneHeart D1	FQ-66006008
245	BeneHeart D1	FQ-64005360	285	BeneHeart D1	FQ-66006009
246	BeneHeart D1	FQ-64005361	286	BeneHeart D1	FQ-66006010
247	BeneHeart D1	FQ-64005362	287	BeneHeart D1	FQ-66006011
248	BeneHeart D1	FQ-64005363	288	BeneHeart D1	FQ-66006012
249	BeneHeart D1	FQ-64005364	289	BeneHeart D1	FQ-66006013
250	BeneHeart D1	FQ-64005365	290	BeneHeart D1	FQ-66006014
251	BeneHeart D1	FQ-64005366	291	BeneHeart D1	FQ-66006015
252	BeneHeart D1	FQ-64005367	292	BeneHeart D1	FQ-66006016
253	BeneHeart D1	FQ-64005368	293	BeneHeart D1	FQ-66006017
254	BeneHeart D1	FQ-64005369	294	BeneHeart D1	FQ-66006018
255	BeneHeart D1	FQ-64005370	295	BeneHeart D1	FQ-66006019
256	BeneHeart D1	FQ-64005371	296	BeneHeart D1	FQ-66006020
257	BeneHeart D1	FQ-64005372	297	BeneHeart D1	FQ-66006021
258	BeneHeart D1	FQ-65005813	298	BeneHeart D1	FQ-66006022
259	BeneHeart D1	FQ-66005975	299	BeneHeart D1	FQ-66006023
260	BeneHeart D1	FQ-66005976	300	BeneHeart D1	FQ-67006026
261	BeneHeart D1	FQ-66005977	301	BeneHeart D1	FQ-67006027
262	BeneHeart D1	FQ-66005978	302	BeneHeart D1	FQ-67006097
263	BeneHeart D1	FQ-66005979	303	BeneHeart D1	FQ-67006098
264	BeneHeart D1	FQ-66005980	304	BeneHeart D1	FQ-67006122
265	BeneHeart D1	FQ-66005981	305	BeneHeart D1	FQ-67006123
266	BeneHeart D1	FQ-66005982	306	BeneHeart D1	FQ-67006124
267	BeneHeart D1	FQ-66005983	307	BeneHeart D1	FQ-67006125
268	BeneHeart D1	FQ-66005984	308	BeneHeart D1	FQ-67006126
269	BeneHeart D1	FQ-66005985	309	BeneHeart D1	FQ-69006480
270	BeneHeart D1	FQ-66005986	310	BeneHeart D1	FQ-69006552
271	BeneHeart D1	FQ-66005987	311	BeneHeart D1	FQ-69006554
272	BeneHeart D1	FQ-66005988	312	BeneHeart D1	FQ-69006555
273	BeneHeart D1	FQ-66005989	313	BeneHeart D1	FQ-69006724
274	BeneHeart D1	FQ-66005990	314	BeneHeart D1	FQ-6A006778
275	BeneHeart D1	FQ-66005991	315	BeneHeart D1	FQ-6A006809
276	BeneHeart D1	FQ-66005992	316	BeneHeart D1	FQ-6A006856
277	BeneHeart D1	FQ-66005993	317	BeneHeart D1	FQ-6A007089
278	BeneHeart D1	FQ-66005994	318	BeneHeart D1	FQ-6A007090
279	BeneHeart D1	FQ-66005995	319	BeneHeart D1	FQ-6A007091
280	BeneHeart D1	FQ-66005996	320	BeneHeart D1	FQ-6B007215

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
321	BeneHeart D1	FQ-6B007216	361	BeneHeart D1	FQ-77010607
322	BeneHeart D1	FQ-6B007375	362	BeneHeart D1	FQ-77010608
323	BeneHeart D1	FQ-71007943	363	BeneHeart D1	FQ-77010609
324	BeneHeart D1	FQ-71007944	364	BeneHeart D1	FQ-77010610
325	BeneHeart D1	FQ-71007945	365	BeneHeart D1	FQ-77010611
326	BeneHeart D1	FQ-71007946	366	BeneHeart D1	FQ-77010850
327	BeneHeart D1	FQ-71008072	367	BeneHeart D1	FQ-77010851
328	BeneHeart D1	FQ-71008133	368	BeneHeart D1	FQ-77010852
329	BeneHeart D1	FQ-72008280	369	BeneHeart D1	FQ-77010853
330	BeneHeart D1	FQ-72008281	370	BeneHeart D1	FQ-77010854
331	BeneHeart D1	FQ-72008282	371	BeneHeart D1	FQ-77010855
332	BeneHeart D1	FQ-72008446	372	BeneHeart D1	FQ-77010856
333	BeneHeart D1	FQ-72008449	373	BeneHeart D1	FQ-77011084
334	BeneHeart D1	FQ-73008878	374	BeneHeart D1	FQ-78011439
335	BeneHeart D1	FQ-74009053	375	BeneHeart D1	FQ-79011656
336	BeneHeart D1	FQ-74009056	376	BeneHeart D1	FQ-7A011935
337	BeneHeart D1	FQ-74009059	377	BeneHeart D1	FQ-7C015098
338	BeneHeart D1	FQ-74009062	378	BeneHeart D1	FQ-81015148
339	BeneHeart D1	FQ-74009072	379	BeneHeart D1	FQ-81015280
340	BeneHeart D1	FQ-74009076	380	BeneHeart D1	FQ-81015281
341	BeneHeart D1	FQ-75009395	381	BeneHeart D1	FQ-81015282
342	BeneHeart D1	FQ-75009396	382	BeneHeart D1	FQ-81015283
343	BeneHeart D1	FQ-76009713	383	BeneHeart D1	FQ-81015457
344	BeneHeart D1	FQ-76009714	384	BeneHeart D1	FQ-83015857
345	BeneHeart D1	FQ-76009715	385	BeneHeart D1	FQ-83015858
346	BeneHeart D1	FQ-77010517	386	BeneHeart D1	FQ-83015859
347	BeneHeart D1	FQ-77010518	387	BeneHeart D1	FQ-83015860
348	BeneHeart D1	FQ-77010519	388	BeneHeart D1	FQ-83015861
349	BeneHeart D1	FQ-77010595	389	BeneHeart D1	FQ-83015862
350	BeneHeart D1	FQ-77010596	390	BeneHeart D1	FQ-83015865
351	BeneHeart D1	FQ-77010597	391	BeneHeart D1	FQ-83015932
352	BeneHeart D1	FQ-77010598	392	BeneHeart D1	FQ-83015933
353	BeneHeart D1	FQ-77010599	393	BeneHeart D1	FQ-84016512
354	BeneHeart D1	FQ-77010600	394	BeneHeart D1	FQ-84016513
355	BeneHeart D1	FQ-77010601	395	BeneHeart D1	FQ-84016514
356	BeneHeart D1	FQ-77010602	396	BeneHeart D1	FQ-84016516
357	BeneHeart D1	FQ-77010603	397	BeneHeart D1	FQ-85016563
358	BeneHeart D1	FQ-77010604	398	BeneHeart D1	FQ-85016564
359	BeneHeart D1	FQ-77010605	399	BeneHeart D1	FQ-85016749
360	BeneHeart D1	FQ-77010606	400	BeneHeart D1	FQ-86017909

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
401	BeneHeart D1	FQ-86017920	441	BeneHeart D1	FQ-88019727
402	BeneHeart D1	FQ-86017921	442	BeneHeart D1	FQ-88019952
403	BeneHeart D1	FQ-86018544	443	BeneHeart D1	FQ-88019953
404	BeneHeart D1	FQ-86018545	444	BeneHeart D1	FQ-88020112
405	BeneHeart D1	FQ-86018546	445	BeneHeart D1	FQ-88020113
406	BeneHeart D1	FQ-86018547	446	BeneHeart D1	FQ-88020114
407	BeneHeart D1	FQ-86018548	447	BeneHeart D1	FQ-88020265
408	BeneHeart D1	FQ-86018549	448	BeneHeart D1	FQ-88020346
409	BeneHeart D1	FQ-86018550	449	BeneHeart D1	FQ-88020347
410	BeneHeart D1	FQ-86018551	450	BeneHeart D1	FQ-88020348
411	BeneHeart D1	FQ-86018552	451	BeneHeart D1	FQ-8A021042
412	BeneHeart D1	FQ-86018553	452	BeneHeart D1	FQ-8B021927
413	BeneHeart D1	FQ-86018554	453	BeneHeart D1	FQ-8B021928
414	BeneHeart D1	FQ-86018555	454	BeneHeart D1	FQ-8B021974
415	BeneHeart D1	FQ-86018556	455	BeneHeart D1	FQ-8B021975
416	BeneHeart D1	FQ-86018557	456	BeneHeart D1	FQ-8B022165
417	BeneHeart D1	FQ-86018558	457	BeneHeart D1	FQ-8B022166
418	BeneHeart D1	FQ-86018559	458	BeneHeart D1	FQ-8C023229
419	BeneHeart D1	FQ-86018560	459	BeneHeart D1	FQ-8C023240
420	BeneHeart D1	FQ-86018561	460	BeneHeart D1	FQ-8C024401
421	BeneHeart D1	FQ-86018562	461	BeneHeart D1	FQ-8C024402
422	BeneHeart D1	FQ-86018563	462	BeneHeart D1	FQ-8C024802
423	BeneHeart D1	FQ-86018564	463	BeneHeart D1	FQ-8C024803
424	BeneHeart D1	FQ-86018565	464	BeneHeart D1	FQ-91024920
425	BeneHeart D1	FQ-86018566	465	BeneHeart D1	FQ-91024921
426	BeneHeart D1	FQ-86018567	466	BeneHeart D1	FQ-91024993
427	BeneHeart D1	FQ-86018568	467	BeneHeart D1	FQ-91024994
428	BeneHeart D1	FQ-86018569	468	BeneHeart D1	FQ-91024995
429	BeneHeart D1	FQ-86018570	469	BeneHeart D1	FQ-91024996
430	BeneHeart D1	FQ-86018571	470	BeneHeart D1	FQ-91025035
431	BeneHeart D1	FQ-86018790	471	BeneHeart D1	FQ-91025275
432	BeneHeart D1	FQ-87018834	472	BeneHeart D1	FQ-91025276
433	BeneHeart D1	FQ-87019050	473	BeneHeart D1	FQ-92025535
434	BeneHeart D1	FQ-87019132	474	BeneHeart D1	FQ-92025536
435	BeneHeart D1	FQ-87019148	475	BeneHeart D1	FQ-92025653
436	BeneHeart D1	FQ-87019390	476	BeneHeart D1	FQ-92025735
437	BeneHeart D1	FQ-87019391	477	BeneHeart D1	FQ-92026099
438	BeneHeart D1	FQ-87019674	478	BeneHeart D1	FQ-93027451
439	BeneHeart D1	FQ-87019675	479	BeneHeart D1	FQ-93027533
440	BeneHeart D1	FQ-87019676	480	BeneHeart D1	FQ-93027561

No.	Commercial Name	Serial Number of products	No.	Commercial Name	Serial Number of products
481	BeneHeart D1	FQ-93027562	501	BeneHeart D1	FQ-96033138
482	BeneHeart D1	FQ-93027563	502	BeneHeart D1	FQ-96033139
483	BeneHeart D1	FQ-94028242	503	BeneHeart D1	FQ-96033140
484	BeneHeart D1	FQ-94028243	504	BeneHeart D1	FQ-96033141
485	BeneHeart D1	FQ-94028495	505	BeneHeart D1	FQ-96033142
486	BeneHeart D1	FQ-94028671	506	BeneHeart D1	FQ-96033143
487	BeneHeart D1	FQ-95030589	507	BeneHeart D1	FQ-97034916
488	BeneHeart D1	FQ-95030590	508	BeneHeart D1	FQ-97034921
489	BeneHeart D1	FQ-95030592	509	BeneHeart D1	FQ-97034922
490	BeneHeart D1	FQ-95030593	510	BeneHeart D1	FQ-97034923
491	BeneHeart D1	FQ-95030597	511	BeneHeart D1	FQ-99039455
492	BeneHeart D1	FQ-95030598	512	BeneHeart D1	FQ-99039456
493	BeneHeart D1	FQ-95030862	513	BeneHeart D1	FQ-99039807
494	BeneHeart D1	FQ-95030863	514	BeneHeart D1	FQ-9A040121
495	BeneHeart D1	FQ-95030864	515	BeneHeart D1	FQ-9A040122
496	BeneHeart D1	FQ-95030968	516	BeneHeart D1	FQ-9A040230
497	BeneHeart D1	FQ-95030969	517	BeneHeart D1	FQ-9A041077
498	BeneHeart D1	FQ-95031584	518	BeneHeart D1	FQ-9B043300
499	BeneHeart D1	FQ-96033136	519	BeneHeart D1	FQ-9B043810
500	BeneHeart D1	FQ-96033137	520	BeneHeart D1	FQ-9C045690
Total: 520					