

Service Type: Product Removal and Replacement			Functional: Safety Related Defect		
MODEL(S)	DESCRIPTION	PART NUMBER (S)	LOT/SERIAL NUMBER(S)	DATE	REV
QRT-DLX	Wheelchair Securement Retractor	Q8-6200	LOT# 58350, LOT ID 0225	30/09/25	3

OVERVIEW

Condition: Retractor may stay unlocked. An unlocked retractor will not properly secure a wheelchair and may place the passenger at a risk of serious injury or death during emergency maneuvers, or a crash.

Correction: Verify Part Number Q8-6200, Lot Number 58350, Lot Identification 0225. Remove and Replace.

Please Report Inspection Results to Q'Strait Technical support at technicalsupport@qstraint.co.uk or +44 (01227)773035 x Ext 428

QRT-DELUXE, Q8-6200 IDENTIFICATION



QRT DELUXE
TWO (2) KNOBS



PART NUMBER
Q8-6200



LOT NUMBER
58350



LOT IDENTIFICATION
0225

ONLY PROCEED IF THE RETRACTOR MATCHES THE PHOTOS ABOVE

STEP	PROCEDURE	ILLUSTRATION / INSTRUCTIONS
1	<u>Verify Retractor Identification per illustration above</u>	Part Number: Q8-6200 Lot Number: 58350 Lot Identification: 0225
2	<u>While awaiting a Replacement Retractor</u> - You may continue using the retractor given you perform a pre-trip inspection and assure the retractor is fully locked. - This can be accomplished by confirming the locking mechanism is engaged and that the webbing and hook do NOT extract out of the retractor. - Once all four (4) wheelchair tiedown retractors are secured to the floor anchorage and wheelchair frame, you can rock the wheelchair back and forth to assure proper locking and tensioning of the retractors.	
3	<u>Remove from Service. Do Not Use!</u> Cut the webbing near label location. Discard retractor. Return hook and label section to Q'Strait.	<p style="text-align: center; font-size: small;">RETURN HOOK AND LABEL SECTION TO Q'STRAIT</p>
4	<u>Retractor Replacement</u> Contact Q'Strait Technical support at technicalsupport@qstraint.co.uk or + 44 (01227)773035 x Ext 428	<u>Provide the Following Information</u> ✓ Customer Name ✓ Confirm Part Number (Q8-6200), Lot Number (58350), and Lot Identification (0225) ✓ Fitting Type: PLI (-L) or Slide N Click (-SC) ✓ Total Quantity of units



FIELD SERVICE BULLETIN

Service Type: Product Removal and Replacement

Functional: Safety Related Defect

MODEL(S)	DESCRIPTION	PART NUMBER (S)	LOT/SERIAL NUMBER(S)	DATE	REV
QRT-DLX	Wheelchair Securement Retractor	Q8-6200	LOT# 58350, LOT ID 0225	30/09/25	3